

# **Toilet Seat Raiser with height adjustment 301701**

## **User's manual**



**MOBILEX A/S**  
Nørskovvej 1  
DK-8660 Skanderborg  
Denmark  
www.mobilex-care.com

## 1. General information

Dear customer,

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

### Intended use of the device

The Mobilex Product is a medical device which is intended to be used by elderly or disabled persons as a seating option and additional support during toilet use. Any other use is prohibited.

By questions please take contact to your dealer or directly to Mobilex A/S.

#### Mobilex A/S

Nørskovvej 1  
DK-8660 Skanderborg  
Denmark

Phone: +45 87 93 22 20  
Telefax: +45 87 93 17 77  
E-Mail: info@mobilex-care.com

## 2. Safety requirements

- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Be aware of the information on the product label.
- Only use the product for the purpose which it is intended for.
- Do not leave or store the product outside.

## 3. Description

### Product description

The Fixed toilet seat raiser is easily fitted directly to the toilet bowl. It attaches to the toilet bowl with adjustable brackets. The seat's height is adjustable in 3 steps, through the attachments at sides of the seat. The sturdy easy to grip arm rests offer support to the user and can be raised to allow side transfer. The armrest can be folded back, and can be easily removed without tools. The seat can be removed for cleaning.

### Assembly description

Height adjustment

- Remove plastic caps located on the sides of the seat
- Loosen the bolts and place the support bars at the desired height.
- Secure the bars at the desired height by tightening the bolts.
- Both support bars should be at the same height
- Insert the caps back in to holes

Attaching the seat to a toilet bowl

- Remove the existing toiled lid and replace it with the Toilet Seat Raiser by inserting the two threaded rods located in the rear bottom of the seat in the holes on the bowl.
- Tighten using two plastic fixings.
- Ensure that the toilet seat raiser is correctly installed before use.

### Transport recommendations

If it is necessary, Toilet Seat Raiser can be easily detached from the toilet and transported

### Accessories

There are no accessories provided for the Toilet Seat Raiser

## 4. Technical data

<b>Toilet Seat Raiser</b>	<b>301701</b>
Total width	43 cm
Total length	49,5 cm
Width between armrests	49 cm
Adjustable height steps	9 cm, 13cm, 17cm
Total weight	6,2 kg
Max. Load	120 kg
Packing dimensions	59 x 52 x 26 cm (L x B x H)

### Product label

The product label is located in the back of the device



Image of the product label (not original)

## 5. Maintenance

Normal dirt on metal and plastic parts can be removed with standard household cleaning agents. Check the specific product information and follow it during cleaning. Use only commercial cleaning products which are suitable for cleaning and disinfection (no solvents or abrasives).

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional

## 6. Most common problems and solutions

Symptoms	possible cause	what to do?
The device isn't in flat position	The height adjustment bars are not on the same height	Fix both bars on the same height
The device isn't stable. It is moving.	The fixing nuts in the back are not tighten.	Check the fixing of the device to a toilet. Tighten the plastic nuts.

## 7. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

## 8. Guarantee terms

Mobilex A/S offers 2 years of guarantee by errors in production. The guarantee is not valid by not-intended use of the product. Parts subject to normal wear are not covered by the guarantee, unless wear is caused by a manufacturing fault. In the event of damage, please notify your dealer. The guarantee does not cover transport cost. The guarantee does not cover injury or other damages eventually related to a malfunction of this product.

## 9. Declaration of Conformity (CE)

The company Mobilex A/S, Nørskovvej 1, 8660 Skanderborg, Denmark declares that the product

### **Toilet Seat Raiser 301701**

is designed in accordance with the technical documentation and made of which conforms to the following basic requirements:

- **EU Council Directive 93/42/EEC for medical equipment Annex I of the June 14, 1993**

Lars Nygaard  
Managing Director  
Mobilex A/S



Skanderborg, the 2006.02.02

.....  
*signature*