

# Fixed toilet seat raiser

## User Manual



**MOBILEX A/S**  
**Noerskovvej 1**  
**8660 Skanderborg**  
**Denmark**

[www.mobilex-care.com](http://www.mobilex-care.com)

## IMPORTANT!

**Before you use the fixed toilet seat raiser, please read this user manual carefully. Pay attention to all instructions, especially those regarding safety, and follow them at all time.**

By queries please contact your dealer or Mobilex A/S direct:

Mobilex A/S  
Noerskovvej 1  
8660 Skanderborg  
Denmark

Telephone: +45 87932220  
Telefax: +45 87931777  
E-Mail: [info@mobilex-care.com](mailto:info@mobilex-care.com)

## 1. General instructions

### Introduction

This operating instruction contains information and instructions regarding the safe and proper use of your fixed toilet seat raiser.

### Proper use

The fixed toilet seat raiser is designed only as a seating option for toilet. Any other use is prohibited.

### Guarantee terms

The producer offers 2 years of guarantee by errors in production. The guarantee is not valid by not-intended use of the product. Parts subject to normal wear are not covered by the guarantee, unless wear is caused by a manufacturing fault. In the event of damage, please notify your dealer. The guarantee does not cover transport cost. The guarantee does not cover injury or other damages eventually related to a malfunction of this product.

### CE marking

The product complies with EU Directive 93/42/EEC for medical equipment.

### Uses symbols

The following symbols are used in these user manual:



**Warning** - This symbol is used for security references, that applies to points that can lead to risk of damages is not followed.



**Reference** – This symbol applies to reference as to how the product should be used.

## 2. Security–General references



- Only use the fixed toilet seat raiser in good technical condition.
- If you detect any crack or damages, please contact your dealer immediately.
- Observe the information on the label. Don't overload the fixed toilet seat raiser.
- Only use the fixed toilet seat raiser for the purpose it is intended for.
- Do not leave or store the product outside.

### 3. Description

The Fixed toilet seat raiser is functional, fitted directly to the toilet bowl. Attaches to the toilet bowl with adjustable brackets. The seat is adjustable in 3 increments, through the attachment at the back of the seat. The sturdy easy to grip arm rests offer support to the user and can be raised to allow side transfer. The armrest can be folded back, and can be easily removed without tools. The seat can be removed for cleaning.

#### Product features:

- Fits most toilets, and remains securely in place.
- With armrests ideal for extra rigidity and stability for sideways transfer.
- Easily fitted to toilet.
- Comes with lid.
- The comfortable plastic seat is made of high density plastic that is easy to clean.

#### Dimension:

Overall width: 43 cm  
Overall depth: 49, 5 cm  
Adjustable seat height: 9cm, 13cm, 17cm  
Width between top of arms: 49cm  
Weight: 6, 2 kg  
Maximum weight of user: 120 kg  
Shipping measurements: 53 cm length, 52 cm width, 26 cm height

#### Product label:

The product label is placed on the back of the seat.



### 4. Usage description

#### Setting up



- Remove the 2 front plastic caps.
- Loosen the screw and place the bar at the desired height.
- Tighten using two nuts.
- Proceed to set the rear bar in the same way. Two bars should be the same height
- Remove the existing toilet lid and replace it by inserting the two threaded rods in the holes.
- Tighten using two plastic fixing.
- Ensure that the toilet seat raiser is correctly installed before use.

## **5. Care and maintenance**

### **Cleaning**

Regularly clean the fixed toilet seat raiser and keep it clean. Use renewal purpose neutral detergent and warm water. Don't use harsh abrasives or bleach based cleaners with may cause damage or discolouration. Dry the parts with a cloth.

For disinfection in hospital environments, please follow the hygiene specialist's recommendations

### **Maintenance**

The fixed toilet seat raiser is designed for minimum maintenance. Periodically check the stability and rigidity of the product. Make sure that the screws are correctly tightened. Worn, damaged or missing components must be replaced immediately by an approved repairer.

### **Deposit the product**

The fixed toilet seat raiser cannot be deposited with normal garbage. It must be delivered to the public station for re-use.