

## **Foldable Shower Chair 302010**

### User's Manual



**MOBILEX A/S**  
Nørskovvej 1  
DK-8660 Skanderborg  
Dänemark  
[www.mobilex-care.com](http://www.mobilex-care.com)

## 1. General Information

Dear customer,

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

### Intended use of the device

The foldable shower chair is medical device which is intended to be used only as a seating option for showers and baths. It's purpose is to help to physically handicapped persons during their hygienic activities. Any other use is prohibited. Don't use this chair as a climbing aid.

By questions please take contact to your dealer or directly to Mobilex A/S.

#### Mobilex A/S

Nørskovvej 1  
DK-8660 Skanderborg  
Dänemark

Telefon: +45 87 93 22 20  
Telefax: +45 87 93 17 77  
E-Mail: info@mobilex-care.com

## 2. Safety requirements

- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb on to the foldable shower chair.
- The foldable shower commode must only be used on stable surface.
- When unfolding the foldable shower chair the user must take care not to squeeze the fingers or other body parts

## 3. Description


### Product description

The Foldable Shower Chair comes with all the standard features of the regular shower chair and is specifically designed for easy carrying and storage. It is designed for individuals who require seating support during showering or while bathing. Made from aluminium tubing it is warm to the touch and corrosion resistant. No tools are required for assembly.

### Assembly description

#### a) Setting up the chair


1. Expand the side frames (A) and (B) towards the front, so there is 90 degrees angle between them and the back frame.
2. Place the tube of the seating frame (B) into the leg holders (C), so the position of chair is stable and locked

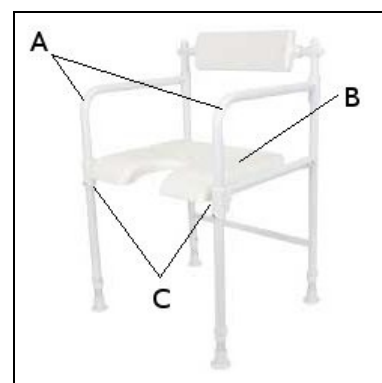

## b) Adjusting the seat height (legs length)

The seat height of the foldable shower chair should be adjusted to suit the user. As a rule, the seat height is correctly adjusted when the user is sitting on the chair and both feet fully touch the ground.

1. Pull the locking clip (A) out of the chair's leg.
2. Adjust the telescopic leg tube to desired height.
3. Push the locking clip (A) in to the locking hole (B).
4. Repeat above steps for the other legs of the chair.

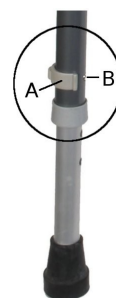

IMPORTANT! Don't adjust the seat of the chair when someone is sitting on it.

## Transport recommendations

The chair can be easily folded for transportation.

## Accessories

There are no accessories provided for foldable shower chair.

## 4. Technical Data:

Foldable Commode Chair	Item no. 302010
Total height	83 - 95 cm
Seat height (adjustable in 2,5 cm steps)	48 - 61 cm
Total width	53 cm
Total length	50 cm
Weight	6,4 kg
Max load	100 kg
Packing dimensions	60 x 16 x 82 cm (L x W x H)

## Product label

The product label is located on main frame

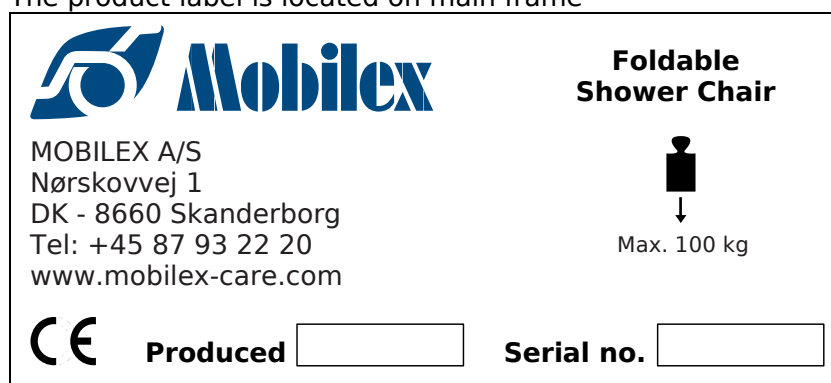


Image of the product label (not original)

## 5. Care & Maintenance

Normal dirt on metal and plastic parts can be removed with standard cleaning agents. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (no solvents or abrasives).

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional.

The foldable shower chair is maintenance free.

## 6. Most common problems and solutions

<i>Symptoms</i>	<i>possible cause</i>	<i>what to do?</i>
Chair is not stable	Legs are adjusted to different heights	Adjust all legs to the same haeight
The seat is not in flat position	The seating frame is not positioned and locked correctly on side supports	Check if the frame is opened correctly and lock seating frame in side supports

## 7. Disposal of the product

The foldable shower chair cannot be deposited with normal garbage. It must be delivered to the public station for re-use.

## 8. Guarantee terms

Mobilex A/S offers 2 years of guarantee by errors in production. The guarantee is not valid by not-intended use of the product. Parts subject to normal wear are not covered by the guarantee, unless wear is caused by a manufacturing fault. In the event of damage, please notify your dealer. The guarantee does not cover transport cost. The guarantee does not cover injury or other damages eventually related to a malfunction of this product.

## 9. Declaration of Conformity (CE)

The company Mobilex A/S, Nørskovvej 1, 8660 Skanderborg, Denmark declares that the product

### **Mobilex Foldable Shower Chair 302010**

is designed in accordance with the technical documentation and made of which conforms to the following basic requirements:

- **EU Council Directive 93/42/EEC for medical equipment Annex I of the June 14, 1993**

Lars Nygaard  
Managing Director  
Mobilex A/S

Skanderborg, am 12.02.2010



.....  
Signature