

# Swivel bather

## User Manual



**MOBILEX A/S**  
**Noerskovvej 1**  
**8660 Skanderborg**  
**Denmark**  
**[www.mobilex-care.com](http://www.mobilex-care.com)**

# IMPORTANT!

**Before you use the swivel bather, please read this user manual carefully. Pay attention to all instructions, especially those regarding safety, and follow them at all time.**

By queries please contact your dealer or Mobilex A/S direct:

Mobilex A/S  
Noerskovvej 1  
8660 Skanderborg  
Denmark

Telephone: +45 87932220  
Telefax: +45 87931777  
E-Mail: [info@mobilex-care.com](mailto:info@mobilex-care.com)

## 1. General instructions

### Introduction

This operating instruction contains information and instructions regarding the safe and proper use of your swivel bather.

### Proper use

The swivel bather is designed only as a seating option for bathroom and showers. Any other use is prohibited. Don't use this chair as a climbing aid.

### Guarantee terms

The producer offers 2 years of guarantee by errors in production. The guarantee is not valid by not-intended use of the product. Parts subject to normal wear are not covered by the guarantee, unless wear is caused by a manufacturing fault. In the event of damage, please notify your dealer. The guarantee does not cover transport cost. The guarantee does not cover injury or other damages eventually related to a malfunction of this product.

### CE marking

The product complies with EU Directive 93/42/EEC for medical equipment.

### Uses symbols

The following symbols are used in this user manual:



**Warning** - This symbol is used for security references, that applies to points that can lead to risk of damages is not followed.



**Reference** – This symbol applies to reference as to how the product should be used.

## 2. Security–General references



- Only use the swivel bather in good technical condition.
- If you detect any crack or damages, please contact your dealer immediately.
- Observe the information on the label. Don't overload the swivel bather.
- Only use the swivel bather for the purpose it is intended for.
- Don't climb on to the swivel bather.

## 3. Description

## Product description

The swivel bather is designed for individuals who require seating support when bathing or showering. It allows the user sits on the seat from the side of the bath and then swivels over the bath for wash or shower.

It is consist of two parts, a base, which is fitted to the bath, and seating unit, which rotates on the base. The base frame has four arms, which rest on the upper part of bath tube. The seat rotates 360 degrees and locks in four positions. The locking lever is located on the right hand side of the seat unit. The seat provides back and arm support and is perfect for people who feel insecure on a bath.

Made from powder coated steel tubing, they are warm to the touch and corrosion resistant.

## Dimensions

Overall width of framework: 72 cm

Overall length of framework: 52 cm

Overall height of framework: 47 cm

Inner bath size: 55 cm to 64 cm

Seat width: 43 cm

Seat depth: 37 cm

Backrest height: 31 cm

Weight of bather: 7 kg

Maximum weight of user: 100 kg

Shipping measurements: 740 mm length, 570 mm width, 390 mm height

## Product label

The product label is placed on the back of the backrest.



## 4. Usage description

### Setting up



- Rest the base frame on the both side of bath tube.
- Ensure that the four arms are firmly placed on the bath.
- Adjust two positioning bolts on the both sides until the rubber ends touch the sides of tube.
- Place the seat unit on the base of frame.
- The socket under the seat unit should be inserted onto the swivel pin of the base frame.
- Make sure that seat unit rotates freely.
- Lock the seat unit, by turning the black scallop handwheel.

### Using the swivel bather



- Put the seat in position to face the side of the bath tube
- Lock the seat by pushing locking lever downwards.
- Now user can be seated on seat unit.
- Unlock the seat by lifting the locking lever and transfer legs of user over the side of the bath tube.
- For extra security while washing, the seat can be locked by locking lever

## **5. Care and maintenance**

### **Cleaning**

Regularly clean the swivel bather and keep it clean. Use renewal purpose neutral detergent and warm water. Don't use harsh abrasives or bleach based cleaners with may cause damage or discolouration. Dry the parts with a cloth.

### **Maintenance**

Check regularly to ensure framework is firmly secured.

### **Deposit the product**

The swivel bather cannot be deposited with normal garbage. It must be delivered to the public station for re-use.