

Shower board

User Manual



MOBILEX A/S
Noerskovvej 1
8660 Skanderborg
Denmark
www.mobilex-care.com

IMPORTANT!

Before you use the shower board, please read this user manual carefully. Pay attention to all instructions, especially those regarding safety, and follow them at all time.

By queries please contact your dealer or Mobilex A/S direct:

Mobilex A/S
Noerskovvej 1
8660 Skanderborg
Denmark

Telephone: +45 87932220
Telefax: +45 87931777
E-Mail: info@mobilex-care.com

1. General instructions

Introduction

This operating instruction contains information and instructions regarding the safe and proper use of your shower board.

Proper use

The shower board is designed only as a seating option for bathtub. Any other use is prohibited. Don't use this board as a climbing aid.

Guarantee terms

The producer offers 2 years of guarantee by errors in production. The guarantee is not valid by not-intended use of the product. Parts subject to normal wear are not covered by the guarantee, unless wear is caused by a manufacturing fault. In the event of damage, please notify your dealer. The guarantee does not cover transport cost. The guarantee does not cover injury or other damages eventually related to a malfunction of this product.

CE marking

The product complies with EU Directive 93/42/EEC for medical equipment.

Uses symbols

The following symbols are used in this user manual:



Warning - This symbol is used for security references, that applies to points that can lead to risk of damages if not followed.



Reference – This symbol applies to reference as to how the product should be used.

2. Security–General references



- Only use the shower board in good technical condition.
- If you detect any crack or damages, please contact your dealer immediately.
- Observe the information on the label. Don't overload the shower board
- Only use the shower board for the purpose it is intended for.
- Don't climb on to the shower board.
- Don't leave or store the product outdoors
- Locate and secure the individual posts into the required positions to suit your bath width and location.

3. Description

Product description

Description:

The shower board allows the user to move safely over to the bath and also provides a comfortable and safe seat. This shower board is portable for storage and travel, manufactured from non corrosive materials. It is capable of being adjusted to suit majority of bathtub widths by setting the individual posts into the required position to cater for the bath position whether free standing or located to the wall.

The shower board has an embossed pattern for slip resistance, and a soap tray. The shower board has a narrow mid-section which allows ease of access for intimate hygiene. Anti-slip rubber pad improves the safety and protects tub surface from scratches.

Technical data

Seat dept:	32 cm
Seat width:	70,5 cm
Height overall:	18 cm
Adjustable inner width:	41 cm – 64,5 cm
Weight:	2,7 kg
Maximum weight of user:	120 kg
Shipping measurements:	73 cm length, 35 cm width, 9 cm height

Product label

The product label is placed on the underside of the board.



4. Usage description

Package content

- 1x board
- 1x handle
- 2x flat side screws for handle
- 4x posts
- 4x screws for posts

Assembly



- The handle is first fitted into the 2 locating holes at one end of the board. The screws are fixed to handle from the underside of the board and can be tighten using a coin.
- The screws for post should be inserted into the slots of board. The posts are fixed to these screws from underside of the board by turning them clockwise.

Using the shower board



- Locate and secure the individual posts into the required positions to suit your bath tube width and location.
- Please ensure there is no sideways movement of the seat and that the post surface is in contact with the bath walls. Both ends of the seat must fully supported by the bath sides.

5. Care and maintenance

Cleaning

Regularly clean the shower board and keep it clean. Use renewal purpose neutral detergent and warm water. Don't use harsh abrasives or bleach based cleaners with may cause damage or discolouration. Dry the parts with a cloth.

Maintenance

The shower board is designed for minimum maintenance. Periodically check the stability and rigidity of the product. Make sure that the screws are correctly tighten. Worn, damaged or missing components must be replaced immediately by an approved repairer.

Deposit the product

The shower board cannot be deposited with normal garbage. It must be delivered to the public station for re-use.