

Wall Mounted Shower Seat

302040-41

User's manual



MOBILEX A/S
Nørskovvej 1
DK-8660 Skanderborg
Denmark
www.mobilex-care.com

1. General information

Dear customer,

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

Intended use of the device

Mobilex Shower Chair is medical device intended only to be used by patients that because of invalidity need support during during washing under the shower. Physically handicapped persons can use it as stable support and to get easy access to all human body parts during washing .The device should be used inside bathrooms only. Bathseat can be maximally loaded with 100 kg. Any other use then intended is strictly prohibited. By questions please take contact to your dealer or directly to Mobilex A/S.

Mobilex A/S

Nørskovvej 1
DK-8660 Skanderborg
Denmark

Phone: +45 87 93 22 20
Telefax: +45 87 93 17 77
E-Mail: info@mobilex-care.com

2. Safety requirements

- Only use the wall mounted shower seat in good technical condition.
- If you detect any crack or damages, please contact your dealer immediately.
- Observe the information on the label. Don't overload the wall mounted shower seat.
- Only use the wall mounted shower seat for the purpose it is intended for.
- Don't climb on to the wall mounted shower seat.
- The wall mounted shower seat must only mounted on sturdy walls.
- Adjust both feet to the same length so they stand stably on the floor surface when chair is opened. The seat surface must be flat then .
- When unfolding the wall mounted shower seat the user must take care not to squeeze the fingers or other body parts between device's elements.

3. Description

Product description

Mobilex Shower Chair is made from powder coated steel tube with a base plate to secure it to the wall, they are warm to the touch and corrosion resistant. The wall mounted shower seat has the seat support rails hinged at the plate allowing it to folded away and locked an upright position. Folds flat against the wall when not in use therefore save space

Assembly description

Shower Chair should be fitted by competent person and secured to a wall capable of supporting the vertical load and leverage affected upon it by the user.

Determine the seat height

- The seat height of the wall mounted shower seat should be adjusted to suit the user.
- As a rule, the seat height is correctly adjusted when the user is sitting on the chair and both feet touch the ground
- Choose a normal chair on which the user can sit comfortable and measure the distant between seat and floor.
- According to it, adjust the legs to desire height.

Adjusting the legs to desire height

- Grasp the horseshoe clip between forefinger and thumb and pull it out.

- Slide the leg to the required position.
- Push the clip firmly back into the hole.
- Set the other legs to same height as described.

Mounting the seat on the wall.

- The wall mounted shower seat must be mounted by a competent person.
- No fixing accessories are supplied.
- The choice of fastening material depends on the condition and types of the wall.
- Select six bolts, washers, and wall plugs.
- Use the mounting plate to mark the position of the six holes.
- Ensure horizontal alignment.
- Drill the holes and insert the wall plugs.
- Place the bracket in the recess between mounting plate and wall.
- Screw the mounting holder tight.
- Make sure that wall mounted shower seat is correctly mounted. To do so fold the wall mounted seat up and down several times .

Usage Description

When folding and unfolding the wall mounted shower seat the user must take care not to squeeze the fingers or other body parts.

- Don't burden the seat surface on one side.
- Grasp the seat at the seat surface as far to the front as possible to fold the seat down.
- When the seat is pulled down the support legs fall automatically to the floor.
- Sit down in the middle of the seat surface.
- After use, move seat up until the seat will be in the upper holding position.

Transport recommendations

The device is dedicated to the place of assembly and should not be transported.

Accessories

There are no accessories provided for the device

4. Technical data

| Mobilex Bathseat | 302040-41 |
|-------------------------------|----------------------------------|
| Overall width | 45 cm |
| Overall length | 45,5 cm |
| Seat width | 44cm |
| Seat depth | 37 cm |
| Adjustable seat height | 43.5 cm - 51cm, 2.5cm increments |
| Weight of the product | 3,5 kg |
| Maximum load | 100 kg |
| Packing dimensions (2 pieces) | 59 x 13 x 48 cm |

Product label

The product label is located on the bottom of the device



Image of the product label (not original)

5. Maintenance & cleaning

The wall mounted shower seat is designed for minimum maintenance. Periodically check the wall plate fittings and the pivot point of the rail are secure. Check regularly to ensure framework is firmly secured.

In case of any doubts concerning technical condition of your bathseat stop using it and contact your dealer.

Normal dirt on metal and plastic parts can be removed with standard household cleaning agents. Check the specific product information and follow it during cleaning. Use only commercial cleaning products which are suitable for cleaning and disinfection (no solvents or abrasives).

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional

6. Most common problems and solutions

| <i>Symptoms</i> | <i>possible cause</i> | <i>what to do?</i> |
|---------------------------------------|--|---|
| The chair doesn't stand stably | Legs adjusted to different heights | Readjust legs to the same height |
| The seat isn't in horizontal position | Legs adjusted to wrong length: - to short (seat falls forward) - to long (seat reclined to the back) | Adjust legs to the length, that allows the seat to lie in horizontal position |
| The chair isn't stable | Wall mountings are loosen | Check the wall assembly and tie all screws. |

7. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

8. Guarantee terms

The producer offers 2 years of guarantee for production failures of the product. The warranty is not valid if damages are caused by not-intended use of the product. Parts which are subjects of normal wear and tear are not covered by the warranty, unless wearing is caused by a manufacturing fault. In the event of damage, please notify your dealer. The warranty does not cover transport cost. The warranty does not cover body injuries or other damages eventually related to a malfunction of this product.

9. Declaration of Conformity (CE)

The company Mobilex A/S, Nørskovvej 1, 8660 Skanderborg, Denmark declares that the product

Mobilex Wall Mounted Shower Seat **302040-41**

is designed in accordance with the technical documentation and made of which conforms to the following basic requirements:

- **EU Council Directive 93/42/EEC for medical equipment Annex I of the June 14, 1993**

Lars Nygaard
Managing Director
Mobilex A/S



Skanderborg, the 01.02.2006

.....
signature