

Wall Mounted Shower Seat

User Manual



MOBILEX A/S
Noerskovvej 1
8660 Skanderborg
Denmark
www.mobilex-care.com

IMPORTANT!

Before you use the wall mounted shower seat, please read this user manual carefully. Pay attention to all instructions, especially those regarding safety, and follow them at all time.

By queries please contact your dealer or Mobilex A/S direct:

Mobilex A/S
Noerskovvej 1
8660 Skanderborg
Denmark

Telephone: +45 87932220
Telefax: +45 87931777
E-Mail: info@mobilex-care.com

1. General instructions

Introduction

This operating instruction contains information and instructions regarding the safe and proper use of your wall mounted seat.

Proper use

The wall mounted shower seat is designed only as a seating option for bathroom and showers. Any other use is prohibited. Don't use this chair as a climbing aid.

It should be fitted by competent person and secured to a wall capable of supporting the vertical load and leverage affected upon it by the user.

Guarantee terms

The producer offers 2 years of guarantee by errors in production. The guarantee is not valid by not-intended use of the product. Parts subject to normal wear are not covered by the guarantee, unless wear is caused by a manufacturing fault. In the event of damage, please notify your dealer. The guarantee does not cover transport cost. The guarantee does not cover injury or other damages eventually related to a malfunction of this product.

CE marking

The product complies with EU Directive 93/42/EEC for medical equipment.

Uses symbols

The following symbols are used in this user manual:



Warning - This symbol is used for security references, that applies to points that can lead to risk of damages is not followed.



Reference – This symbol applies to reference as to how the product should be used.

2. Security–General references



- Only use the wall mounted shower seat in good technical condition.
- If you detect any crack or damages, please contact your dealer immediately.
- Observe the information on the label. Don't overload the wall mounted shower seat.
- Only use the wall mounted shower seat for the purpose it is intended for.
- Don't climb on to the wall mounted shower seat.
- The wall mounted shower seat must only mounted on sturdy walls.
- Adjust both feet to the same length.
- When unfolding the wall mounted shower seat the user must take care not to squeeze the fingers or other body parts.

3. Description

Product description


The wall mounted shower seat is designed for individuals who require seating support when bathing or showering. It is made from powder coated steel tube with a base plate to secure it to the wall, they are warm to the touch and corrosion resistant. The wall mounted shower seat has the seat support rails hinged at the plate allowing it to be folded away and locked in an upright position. It folds flat against the wall when not in use therefore saving space.

Technical data

Overall width:	45 cm
Overall depth:	45,5 cm
Seat width:	44 cm
Seat depth:	37 cm
Adjustable seat height:	43.5 cm – 51cm, 2.5cm increments
Weight:	3,5 kg
Maximum weight of user:	100 kg
Shipping measurements:	59 cm length, 13 cm width, 48 cm height

Product label

The product label is placed on the back of the backrest.

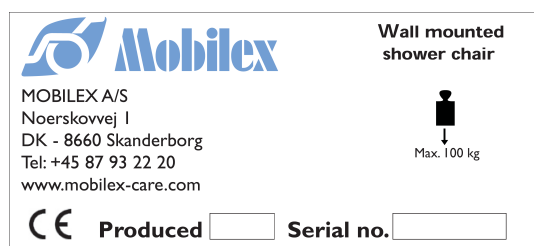

4. Mounting

Determine the seat height



- The seat height of the wall mounted shower seat should be adjusted to suit the user.
- As a rule, the seat height is correctly adjusted when the user is sitting on the chair and both feet touch the ground.
- Choose a normal chair on which the user can sit comfortably and measure the distance between seat and floor.
- According to it, adjust the legs to the desired height.

Adjusting the legs to the desired height

- Grasp the horseshoe clip between forefinger and thumb and pull it out.
- Slide the leg to the required position.
- Push the clip firmly back into the hole.
- Set the other legs to the same height as described.

Mounting the seat on the wall.



- The wall mounted shower seat must be mounted by a competent person.
- No fixing accessories are supplied.
- The choice of fastening material depends on the condition and types of the wall.



- Select six bolts, washers, and wall plugs.
- Use the mounting plate to mark the position of the six holes.
- Ensure horizontal alignment.
- Drill the holes and insert the wall plugs.
- Place the bracket in the recess between mounting plate and wall.
- Screw the mounting holder tight.
- Make sure that wall mounted shower seat is correctly mounted. To do so fold the wall mounted seat up and down several times .

5. Usage description



- When folding and unfolding the wall mounted shower seat the user must take care not to squeeze the fingers or other body parts.
- Don't burden the seat surface on one side.

Use



- Grasp the seat at the seat surface as far to the front as possible to fold the seat down.
- When the seat is pulled down the support legs fall automatically to the floor.
- Sit down in the middle of the seat surface.
- After use, move seat up until the seat will be in the upper holding position.

6. Care and maintenance

Cleaning

Regularly clean the wall mounted shower seat and keep it clean. Use renewal purpose neutral detergent and warm water. Don't use harsh abrasives or bleach based cleaners with may cause damage or discolouration. Dry the parts with a cloth.

Maintenance

The wall mounted shower seat is designed for minimum maintenance.

Periodically check the wall plate fittings and the pivot point of the rail are secure.

Check regularly to ensure framework is firmly secured.

Deposit the product

The wall mounted shower seat cannot be deposited with normal garbage. It must be delivered to the public station for re-use.