

Bath bench

User Manual



CE

MOBILEX A/S
Noerskovvej 1
8660 Skanderborg
Denmark
www.mobilex-care.com

IMPORTANT!

Before you use the bath bench, please read this user manual carefully. Pay attention to all instructions, especially those regarding safety, and follow them at all time.

By queries please contact your dealer or Mobilex A/S direct:

Mobilex A/S
Noerskovvej 1
8660 Skanderborg
Denmark

Telephone: +45 87932220
Telefax: +45 87931777
E-Mail: [**info@mobilex-care.com**](mailto:info@mobilex-care.com)

1. General instructions

Introduction

This operating instruction contains information and instructions regarding the safe and proper use of your bath bench.

Proper use

The bath bench is designed only as a seating option for bathroom and showers. Any other use is prohibited. Don't use this chair as a climbing aid.

Guarantee terms

The producer offers 2 years of guarantee by errors in production. The guarantee is not valid by not-intended use of the product. Parts subject to normal wear are not covered by the guarantee, unless wear is caused by a manufacturing fault. In the event of damage, please notify your dealer. The guarantee does not cover transport cost. The guarantee does not cover injury or other damages eventually related to a malfunction of this product.

CE marking

The product complies with EU Directive 93/42/EEC for medical equipment.

Uses symbols

The following symbols are used in these user manual:



Warning - This symbol is used for security references, that applies to points that can lead to risk of damages is not followed.



Reference – This symbol applies to reference as to how the product should be used.

2. Security–General references



- Only use the bath bench in good technical condition.
- If you detect any crack or damages, please contact your dealer immediately.
- Observe the information on the label. Don't overload the foldable bath bench.
- Only use the bath bench for the purpose it is intended for.
- Don't climb on to the bath bench.
- The bath bench must only be used on stable surface.
- When adjusting the bath bench the user must take care not to squeeze the fingers or other body parts

3. Description

Product description


The Bath Bench is designed for individuals who require seating support when bathing or showering. Made from aluminium tubing they are warm to the touch and corrosion resistant. No tools are required for assembly.

Technical data:

	Seat width 42 cm	Seat width 47 cm
Overall height:	600 – 770 mm	600 – 770 mm
Seat width:	420 mm	470 mm
Seat dept:	335 mm	335 mm
Seat height:	420 – 590 mm, 25mm steps	420 – 590 mm
Weight of chair:	2,8 kg	3,0 kg
Maximum weight of user:	130 kg	130 kg

Product label

The product label is placed on the back.

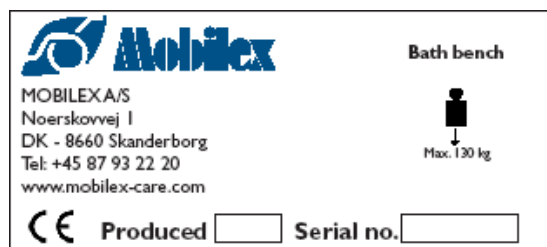

4. Usage description


Adjusting the legs to desired height



- The seat height of the bath bench should be adjusted to suit the user.
- As a rule, the seat height is correctly adjusted when the user is sitting on the chair and both feet touch the ground.
- Don't adjust the seat of the chair when someone is sitting on it.



- Grasp the horseshoe clip between forefinger and thumb and pull it out.
- Slide the leg to the required position.
- Push the clip firmly back into the hole.
- Set the other legs to same height as described.

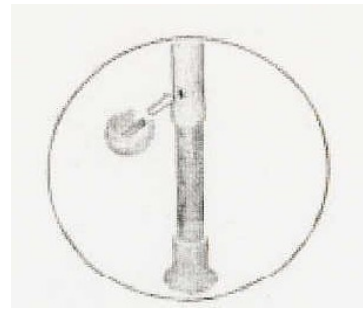

Care and maintenance

Cleaning

Regularly clean the bath bench and keep it clean. Use renewal purpose neutral detergent and warm water. Don't use harsh abrasives or bleach based cleaners with may cause damage or discolouration. Dry the parts with a cloth.

Maintenance

The bath bench is maintenance free.

Deposit the product

The bath bench cannot be deposited with normal garbage. It must be delivered to the public station for re-use.