


# Alligator Ramps

## CH-Rampe

### User Manual

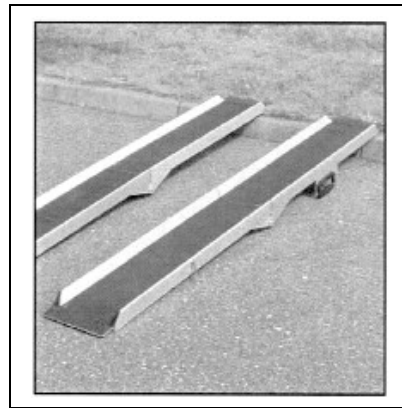

**Contact:**

**Mobilex A/S**

**Nørskovvej 1**

**8660 Skanderborg**

**Danmark**

<a href="#">CH-Ramps List of contents.....</a>	<a href="#">2</a>
<a href="#">1 CH-Ramps List of contents.....</a>	<a href="#">3</a>
<a href="#">2 CH-Ramps Security reference.....</a>	<a href="#">4</a>
<a href="#">2.1 Security – General references.....</a>	<a href="#">4</a>
<a href="#">3 CH-Ramps Purpose description.....</a>	<a href="#">5</a>
<a href="#">4 CH-Ramper Indication.....</a>	<a href="#">5</a>
<a href="#">5 CH-Ramps Declaration of conformity.....</a>	<a href="#">6</a>
<a href="#">6 CH-Ramps Product description.....</a>	<a href="#">7</a>
<a href="#">6.1 Produktdescription.....</a>	<a href="#">7</a>
<a href="#">7 CH-Ramps Delivery description.....</a>	<a href="#">7</a>
<a href="#">8 CH-Ramps Usage description.....</a>	<a href="#">7</a>
<a href="#">9 CH-Ramper Transport description.....</a>	<a href="#">8</a>
<a href="#">10 CH-Ramps Technical data.....</a>	<a href="#">8</a>
<a href="#">11 CH-Ramps Product label.....</a>	<a href="#">9</a>
<a href="#">12 CH-Ramps Cleaning description.....</a>	<a href="#">9</a>
<a href="#">13 CH-Ramps Maintenance.....</a>	<a href="#">9</a>
<a href="#">14 CH-Ramps How to deposit the product.....</a>	<a href="#">9</a>
<a href="#">15 CH-Ramps Guarantee.....</a>	<a href="#">10</a>
<a href="#">16 CH-Ramps Notes.....</a>	<a href="#">10</a>

This user manual describes how the Alligator CH ramps are used and maintained.

By queries please contact your dealer or Mobilex direct:

**Mobilex A/S**

**Nørskovvej 1**

**8660 Skanderborg**

**Danmark**

Telephone: +45 87932220

Telefax.....: +45 87931777

E-mail .....: [info@mobilex-care.com](mailto:info@mobilex-care.com)

### Used symbols:

In the user manual the following symbols are used:

#### **Warning**

This symbol is used for security references, that applies to points that can lead to risk of damages is not followed.

#### **Reference**

This symbol applies to a reference as to how the product should be used.

### 2.1 Security – General references

Make sure that this user manual is read by all persons using the ramp. Damages, caused by the fact that the user manual has not been followed, will not be accepted.

#### **Limitations for usage**

- CH-Ramps must not be loaded with more than 300 kg
- The ramps can maximum be used at inclinations like:

**CH-180                      35 cm**

#### **Danger for accidents**

- Only use the ramp for the purpose it is intended for.
- Only use ramps in good technical condition.
- Avoid making constructive changes on the ramp, unless you have the producers written accept.
- When unfolding the ramp the user must take care not to squeeze the fingers or other body parts.
- The ramp must only be used on a stable surface.

### **3 CH-Ramps**

### **Purpose description**

Mobilex Ramps are transportable ramps intended to move patients in wheelchairs or scooters. They enable the passage of elevated points with or without the assistance of a helper (f.ex stairs). Mobilex ramps thereby enables patients, sitting in a wheelchair because of invalidity, to enter or leave a building and thus serving a fundamental need. The ramps can be used as well inside as outside. Alligator ramps of the type CH can maximum be loaded with 300 kg.

### **4 CH-Ramper**

### **Indication**

Mobilex ramps are intended for use by patients that because of invalidity are sitting in a wheelchair. They make it possible for the patients to pass elevated points with or without the assistance of a helper.

## 5 CH-Ramps

## Declaration of conformity

### Producer:

Mobilex A/S  
Nørskovvej 1  
8660 Skanderborg  
Danmark

erklærer hermed, at produktet: Alligator Ramps Type CH

are designed and produced in accordance with the technical documentation and according to the demands of:

- Guidelines 93/42/EWG for medical products Part I from 14 Juni 1993

### Responsibility:

**Company:** Mobilex A/S, Noerskovvej 1 24, 8660 Skanderborg, Denmark

**Title:** Managing director

**Name:** Lars Nygaard

Skanderborg 12/8-2004

-----  
Dato




-----  
Underskrift

## 6 CH-Ramps

## Product description

### 6.1 Produktdescription

Alligator ramps, Type DF are made from aluminium.

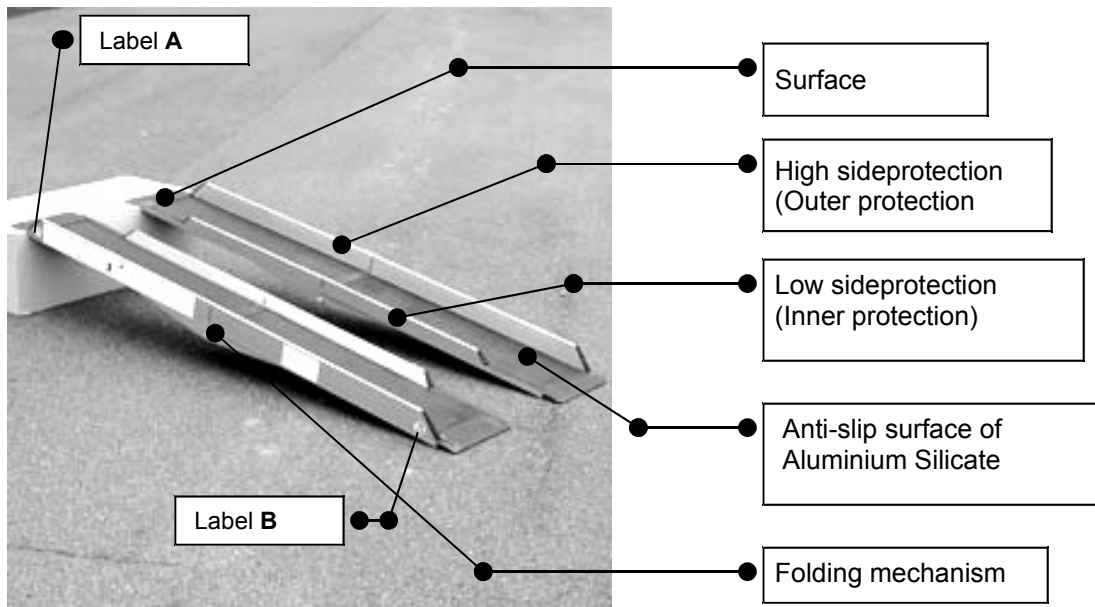

## 7 CH-Ramps

## Delivery description

CH-Ramps are supplied in cartons:

The contents are:

- 1 Alligator-Ramp Type DF
- 1 User manual
- 1 set of boxes

## 8 CH-Ramps

## Usage description



CH-Ramps are fitted with a label **A** and a label **B** (see picture in section 6).

- Label **A** is placed on the upper side of the ramp (the side placed on the hindrance)
- Label **B** is placed on the lower side of the ramp (the side placed at the ground)
- The ramps must be placed so that the side protection is outwards (see picture in section 6).
- The ramps must be placed on a stable surface.
- Check that the ramps are stable before driving on them.
- Use only the ramps for height differences up to 35 cm.

## 9 CH-Ramper

## Transport description

Alligator ramps type CH can for be folded for transport and locked with a hinge.

## 10 CH-Ramps

## Technical data


<b>Alligator type CH-180</b>	
Length:	2 x 180 cm
Width:	2 x 23,5 cm
Surface:	2 x 21 cm
Height of sideprotections:	Outer protection: 6 cm / Inner: 3,3 cm
Max. elevation / Height difference:	12° / 35 cm
Weight total:	13 kg
Max. load:	300 kg
Remarks:	Used as a pair.

## 11 CH-Ramps

## Product label

The product label is placed on the outside of the ramp:

The product label:

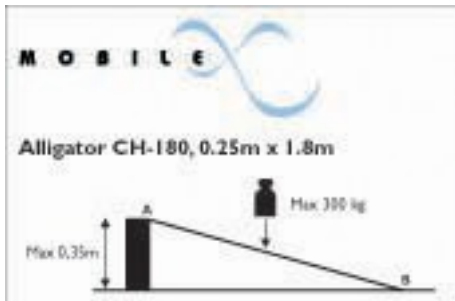

## 12 CH-Ramps

## Cleaning description

- Lighter dirt and dust can be removed with a wet cloth
- Larger parts can be removed with a non-corroding, mild washing agent.

## 13 CH-Ramps

## Maintenance

The ramps are maintenance free.

## 14 CH-Ramps

## How to deposit the product

The ramps cannot be deposited with normal garbage.  
They must be delivered to the public station for re-use.

**15 CH-Ramps**

**Guarantee**

The producer offers 2 years of guarantee by errors in production. The guarantee is not valid by not-intended use of the product.

**16 CH-Ramps**

**Notes**

---

---

---

---

---

---

---

---

---

---

---