

# "Alligator" Ramps

## DF - Ramps

### User's manual



**MOBILEX A/S**  
Nørskovvej 1  
DK-8660 Skanderborg  
Denmark  
[www.mobilex-care.com](http://www.mobilex-care.com)

## 1. General information

Dear customer,

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

### Intended use of the device

The Mobilex Alligator Ramps are medical devices which is intended to be used only to assist patients in wheelchairs or scooters. They enable the passage of elevated points (f.e. Stairs) with or without the assistance of a helper. Mobilex Ramps thereby enables patients, sitting in a wheelchair because of invalidity, to enter or leave buildings or cars, buses etc. The ramps can be used as well inside as outside.

Any other use is prohibited

By questions please take contact to your dealer or directly to Mobilex A/S.

### Mobilex A/S

Nørskovvej 1  
DK-8660 Skanderborg  
Denmark

Phone: +45 87 93 22 20  
Telefax: +45 87 93 17 77  
E-Mail: info@mobilex-care.com

## 2. Safety requirements

- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Be aware of the information on the product label.
- Make sure that this user manual is read by all persons using the ramp. Damages, caused by the fact that the user manual has not been followed, will not be accepted.
- Ramps must not be loaded with more than the "max load" described in Technical Data and on the label.
- The ramps can be used at maximum inclination of:  
**DF-180      35 cm**  
**DF-240      50 cm**  
**DF-300      60 cm**
- Only use the ramp for the purpose it is intended for.
- Avoid making constructive changes on the ramp, unless you have the producers written accept.
- The ramp must only be used on a stable surface.
- Be careful during folding or unfolding of the ramp. There is a risk of finger squeeze or cut between parts of the ramp.

## 3. Description


### Product description

Mobilex DF-Ramps are foldable platforms made of aluminium. The surface is covered with special material for anti-slipping purposes. To make handling and transportation easier ramps are divided in to two foldable halves than can be assembled together for usage.

SF-Ramps are supplied in cartons. The contents are:

- 1 Ramp
- 1 User manual

## Set up & Function description

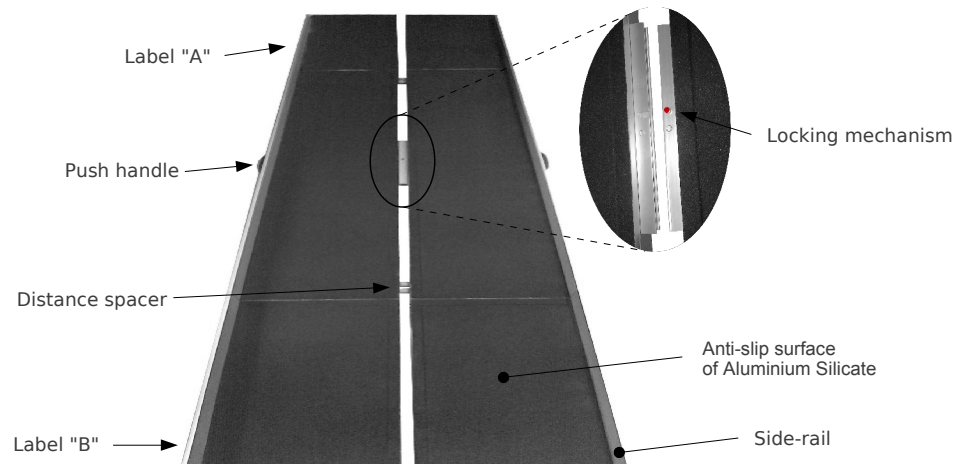

- Ramps are marked with label **A** and label **B**.
- Label **A** is placed on the upper side of the ramp (the side placed on the hindrance).
- Label **B** is placed on the lower side of the ramp (the side placed at the ground).
- The ramp parts must be fitted together with the red springs.
- The ramp must be placed so that the high side protection is outwards.
- The ramp must be placed on a stable surface.
- Check that the ramp is stable & ramp halves are connected before driving on it.

### Transport recommendations

Alligator ramps type DF can be disassembled in to two halves and folded (with hinge) for transport. Always assure that ramp during transportation is safely secured against unintended movements.

### Accessories

There are no accessories provided for Alligator Ramps.

## 4. Technical data

Mobilex „Alligator“ Ramps, DF type	DF-180	DF-240	DF-300
Length	2x180 cm	2x240 cm	2x300 cm
Width (unfolded)	79 cm	79 cm	79 cm
Surface width	75 cm	75 cm	75 cm
Height of side protections	6 cm (inner height)	6 cm (inner height)	6 cm (inner height)
Max. elevation / Height difference	12° / 35 cm	12° / 50 cm	12° / 60 cm
Total weight	16 kg	22 kg	27,2 kg
Max. load	300 kg	300 kg	300 kg

## Product label

The product label is located on one of the ramp sides

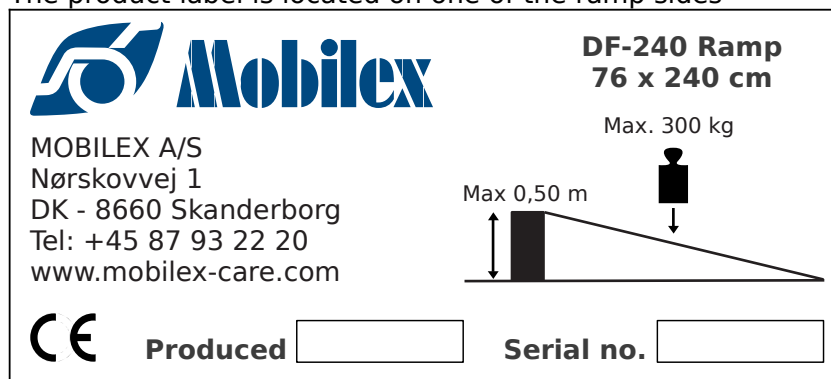


Image of the product label (not original)

## 5. Maintenance & Cleaning

### Maintenance

Ramps are basically maintenance free. Although it is recommended to check always before use if the anti-slip surface isn't dirty or torn and if hinge connection between two ramp halves is rigid and strong. In case of any doubts concerning technical condition of your ramp stop using it and contact your dealer.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional

### Cleaning

- Lighter dirt and dust can be removed with a wet cloth.
- Heavier types of dirt can be removed with a non-corroding, mild washing agent.
- You can also use garden hose to clean the ramp.

## 6. Most common problems and solutions

Symptoms	possible cause	what to do?
Ramp is not stable	The surface under the ramp is not flat	Check the position of the ramp. Put it on flat, smooth surface
Ramp is not stable	Ramp is putted on the surface upside down	Check if the A label is in the upper end and B label in the lower end of the ramp. Correct position if needed.
Ramp doesn't open	The lock strap is not loosen	Loosen the lock strap
Ramp halves are not parallel	The ramp halves are not correctly secured with red locks	Reposition ramp halves and connect them together with red locks

## 7. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

## 8. Guarantee terms

Mobilex A/S offers 2 years of guarantee by errors in production. The guarantee is not valid by not-intended use of the product. Parts subject to normal wear are not covered by the guarantee, unless wear is caused by a manufacturing fault. In the event of damage, please notify your dealer. The guarantee does not cover transport cost. The guarantee does not cover injury or other damages eventually related to a malfunction of this product.

## 9. Declaration of Conformity (CE)

The company Mobilex A/S, Nørskovvej 1, 8660 Skanderborg, Denmark declares that the product


### **Alligator Ramp - DF series**

is designed in accordance with the technical documentation and made of which conforms to the following basic requirements:

- **EU Council Directive 93/42/EEC for medical equipment**  
Annex I of the June 14, 1993

Lars Nygaard  
Managing Director  
Mobilex A/S

Skanderborg, the 30.06.2009



.....  
*signature*