
## CH ramps

Art. CH-180

**Montage- und Gebrauchsanleitung**  
**Montage- og brugervejledning**  
**User Manual**  
**Manuale d'uso e manutenzione**

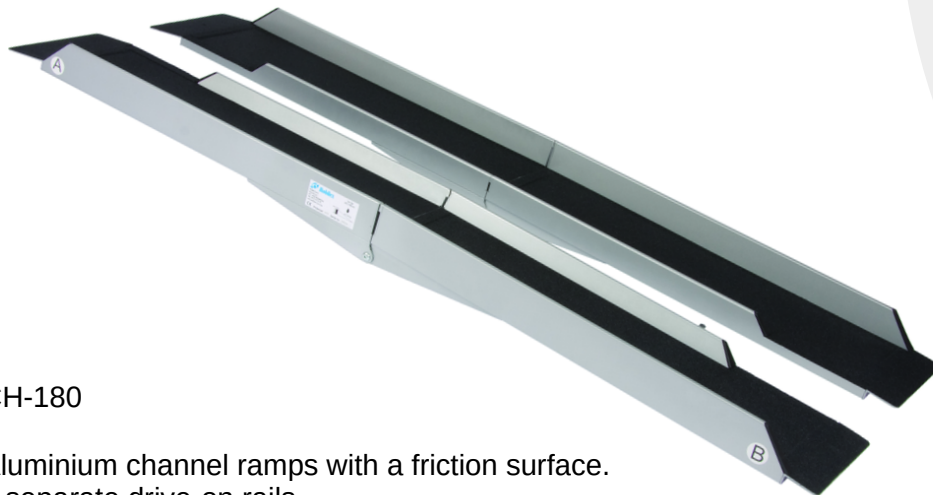CH-180

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## 1. Introduction

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

### Indications

Ramps are indicated for people with mobility limitations who due to problems with walking, standing or sitting are forced to use mobility devices such as wheelchairs, scooters, rollators etc. Ramps are indicated for elimination of architectural obstacles by smooth connection of the level differences that a wheelchair user has to overcome and thus they allow the wheelchair user to reach places normally inaccessible to him/her. The ramp is intended for use independently or with the help of an attendant. Using the ramp for purposes other than those indicated above, is prohibited.

### Contraindications

Contraindications cannot be tied to one, specific clinical picture. The choice of a ramp type and length that is appropriate for a given must be always made with the help of a specialist physician or physiotherapist.

By questions please take contact to your dealer or directly to Mobilex A/S.

<b>Mobilex A/S</b> Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 E-Mail: info@mobilex.dk
--	---

## 2. Safety requirements

- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Be aware of the information on the product label.
- Make sure that this user manual is read by all persons using the ramp. Damages, caused by the fact that the user manual has not been followed, will not be accepted.
- Ramps must not be loaded with more than the "max load" described in Technical Data and on the label.
- The ramps can be used at maximum inclination of:  
**CH-180: 35 cm**
- Only use the ramp for the purpose it is intended for.
- Avoid making constructive changes on the ramp, unless you have the producers written accept.
- The ramp must only be used on a stable surface.
- Be careful during folding or unfolding of the ramp. There is a risk of finger squeeze or cut between parts of the ramp.

## 3. Description


### Product description

Mobilex CH-Ramp set consists of 2 separate rails made of aluminium. The surface is covered with special anti slipping material. To make handling and transportation easier ramps can be folded with hinge to for transport.

CH-Ramps are supplied in 2 cartons.

The contents are:

- ✓ 2 Ramp Rails
- ✓ 1 User manual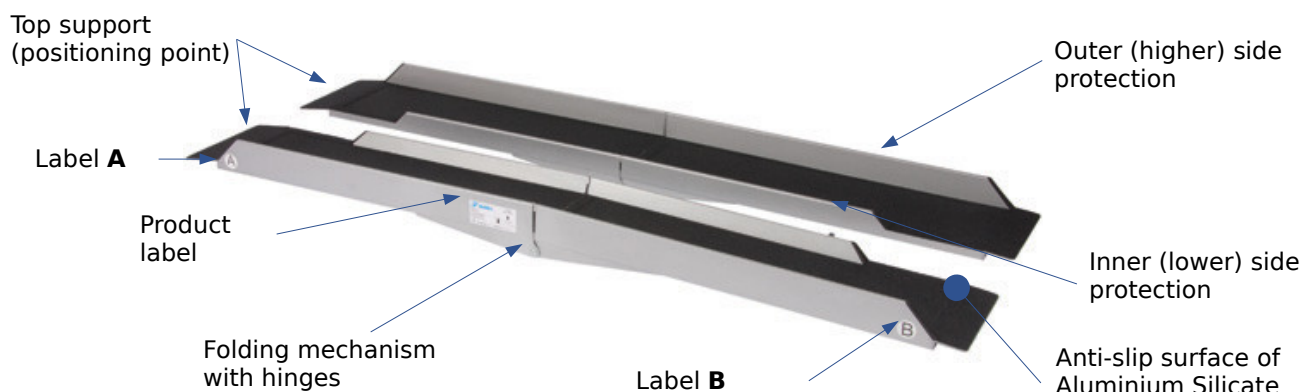

### Set Up & Function description

- Ramps are marked with label **A** and label **B**.
- Label **A** is placed on the upper side of the ramp (the side placed on the hindrance).
- Label **B** is placed on the lower side of the ramp (the side placed at the ground).
- The ramp halves must always be positioned in parallel position.
- The ramp must be placed so that the higher side protection is outwards.
- The ramp must be placed on a stable surface.
- Check that the ramp is stable before driving on it.

### Transport recommendations

Alligator ramps type CH can be folded (with hinge) and locked for transport. Always assure that ramp during transportation is safely secured against unintended movements.

### Accessories

There are no accessories provided for Alligator Ramps.

### 4. Technical data

Mobilex CH-Ramps	CH-180
Length	180 cm
Width (per rail)	23,5 cm
Surface width	21 cm
Height of side protect. (ext./int.)	6,0 cm / 3,3 cm
Max. Elev. / height	12° / 36 cm
Weight (1 pair)	13 kg
Max. load	300 kg
Folded size (LxWxH)	93 x 27 x 21 cm

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Batch no/serial no
	CE-marked product		Item Number
	To be used indoor and outdoor		Medical Device
			Max Elevation / height

Overview icon meaning

### Product label

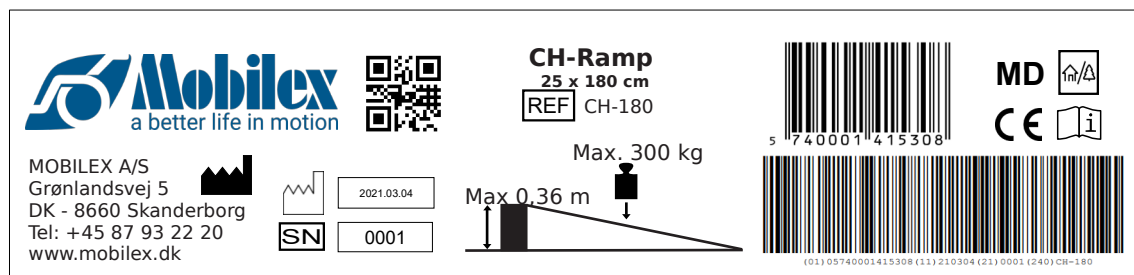
Image of the product label for CH-180 ramp

## 5. Maintenance and cleaning

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional

- Lighter dirt and dust can be removed with a wet cloth.
- Heavier types of dirt can be removed with a non-corroding, mild washing agent.
- You can also use garden hose to clean the ramp.

**Recommended service life:** Normally 10 years depending on use.

### **Warning**

 Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

### **Storage**

We recommend storage in a dry place.

### **Reuse**

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

## 6. Most common problems and solutions

<i>Symptoms</i>	<i>possible cause</i>	<i>what to do?</i>
Ramp is not stable	The surface under the ramp is not flat	Check the position of the ramp. Put it on flat, smooth surface
Ramp is not stable	Ramp is putted on the surface upside down	Check if the A label is in the upper end and B label in the lower end of the ramp. Correct position if needed.
Ramp doesn't open	The lock strap is not loosen	Loosen the lock strap

## 7. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

## 8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,

**Hereby declare under our sole responsibility as a legal manufacturer** that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

**PRODUCT LIST**

**CH ramp - item no. CH-180**

**ACCESSORIES LIST**

Item no.	Accessories item no.
CH-180	No accessories available

Harmonized norms used during conformity estimation:

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**In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /**

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