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mobilex.dk/medias/2021-09/ce_toilet_og_badestol_kakadu_tilt_302024_25_8560.pdf

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

Use is indicated for people with limited standing ability and reduced or no functional ability of the lower extremities. The commode wheelchair is a medical device intended to be used only as seating support by individuals who require such help during mostly independent bathing, showering and defecating (the bathroom must be compatible). It is also a transport option intended to transport patients from and to bathrooms or toilets. The device serves a fundamental function and makes the life of both the physically challenged user and the assistant much easier. The device should only be used indoors. Any other use is prohibited.

Indications

The commode and shower chair "Kakadu" with 5" wheels and the version with 24" plastic wheels are indicated for people with mobility limitations who have difficulty walking, standing, or sitting without stable support, possibly as a result of structural and/or functional damage to the lower limbs (e.g. amputation, consequences of injuries, musculoskeletal/neuromusculoskeletal movement disorders). These assistive products are dedicated to people who, due to their disability, cannot perform their daily hygiene activities in regular shower cubicles or bathtubs and who are unable to use regular toilets, even after modifications.

In addition, for the standard version with four 5" wheels: Inability to propel a wheelchair (e.g. in case of limited strength and grip function or lack of arm/hand coordination).

For the version with the optional 24" wheels: If the strength and grip function of the arms/hands are still sufficiently preserved.

Contraindications

The Commode and shower chair "KAKADU" should not be used by persons who cannot maintain a stable sitting position.

For the commode and shower chair "KAKADU" and for the version with the optional 24" wheel set:

Users with cognitive impairment must be monitored during use of the device and it is recommended that an individual assessment of the user's health condition is carried out before use. The selection of the orthopaedic device that is suitable for a given medical condition must always be made with the help of a specialist doctor or physiotherapist. In addition, you must comply with the safety information in chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S	Phone:	+45 87 93 22 20
Grønlandsvej 5	Telefax:	+45 87 93 17 77
DK-8660 Skanderborg	Email:	info@mobilex.dk
Denmark		

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- Use caution when assembling or adjusting any parts of the device.
- The device must not be loaded with more weight than indicated in Technical data.
- All wheels MUST be in contact with the floor at ALL TIMES during use. This will ensure that the device is properly balanced and should decrease the risk of accidents.
- The device must only be used on a stable surface.
- When using the device in a stationary position, the brakes MUST be engaged.
- Do not stand on footrests, and do not sit on the edge of the seat with the legs on the footrests
- The device is only to be used indoors and on flat surfaces without any obstacles.
- Always maintain a stable sitting position when using this device and users should not be left on the device unattended. Remove the seat cover only when chair is used for going to the toilet..
- If the seat cover is placed over the seat hole, it is absolutely necessary to also place the bucket WITH THE LID under the seat.

WARNING! Because of the tilting function the device has a specially designed front frame. Do NOT lean too far forward and do not sit on the edge of the seat pad with your feet placed on the footrests. By moving the center of gravity too much to the front with all your weight located on the chair's front frame, you may cause the device to tip over. It may result in property damage or personal injury. If you want to lean forward to reach some objects, or if you want to move your body to the front edge of the seat for a transfer, **ALWAYS** put your feet on the floor first, so that a part of your body weight is removed from the device. The brakes must always be engaged in such a situation.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Main frame with seat;
- ✓ 1 Detachable backrest frame with armrests;
- ✓ 2 Detachable footrests;
- ✓ 4 Detachable wheel assemblies consisting of wheel, brake and vertical assembly tube;
- ✓ 4 Bolts and 4 nuts for wheel assembly;
- ✓ 1 Sanitary bucket with a lid;
- ✓ 1 Set of tools for mounting;
- ✓ 1 User manual;
- ✓ 24" plastic wheels (optional set): 24" plastic wheels, quick release and brake kit;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

Kakadu commode and shower chair comes with all the standard features of the regular commode and shower chair: replaceable sanitary bucket, parking brakes, detachable footrests, elevable armrests etc. The product is made in materials that is corrosion and disinfectant resistant. The frame is made of powder-coated aluminium tube. It is additionally equipped with a tilt function.

The device consists of the following elements:

1. Headrest
2. Seat tilt adjustment lever
3. Back frame assembly knob
4. Soft PU seat
5. Removable seat cover
6. Detachable sanitary bucket
7. Turning wheel with pedal brakes
8. Soft PU backrest
9. Elevable armrest
10. Footrest
11. Footrest detaching lever
12. Adjustment bolt
13. Lifiable footplate

Assembly description


- Put the main frame upside down on the floor. You'll see four holes at the ends of vertical frame tubes.
- Insert the wheel assemblies into the tubes and fix them in desired setting using provided bolts, as described in **Function description** section. Check if they are attached correctly by trying to pull them out. If they slide around, check that the bolt went through both the frame and wheel assembly and tighten the bolt again.
- Put the frame on its wheels and lock the wheel brakes by pushing them down with the tip of a foot to make further assembly safe.
- Insert the backrest tubes into holes on the rear end of main frame. Tighten the knobs on both sides of the chair and check that the backrest is stable.
- Put the vertical bar of the headrest into the clamp located at the middle top of the backrest. Adjust the headrest to the desired height, and tighten the bolt on the clamp.
- Put the lid on the sanitary bucket and slide it into the guide rails under the seat from the rear of the chair. Once the bucket is in place, place the PU seat on the frame by pressing the seat clamps on the frame and add the removable seat cover on the PU seat. Seat and seat cover can be replaced.
- Put the top end of the footrest into the hole in the front of the main frame, and turn the footrest towards the front of the frame until it's locked. Unfold the footplate and repeat for the other footrest.

24" Rear wheels (accessory)


- Detach the 2 rear standard wheels in an opposite way to the previous assembly instruction.
- There are 3 height settings which consist of 2 adjacent holes. Match the wheel assembly with the 2 holes of a desired setting and make sure the bearing housing is on the rear side of the frame (Pic. 1). Fix the assembly to the tube by putting bolts through the assembly and tube, and tightening them together with a washer and a nut. Repeat above steps for the other side of the chair.


- Press and hold the quick release button located in the middle of the 24" wheel. Insert the wheel axle into the housing on the assembly plate and release the button (Pic. 2). Always check if the wheels are attached correctly by trying to pull them out. After wheel installation, put on the caps for the quick release buttons (Pic. 3).
- To take the wheel out remove the cap press the quick release button and pull the wheel out of its housing.
- After the assembly of the 24" wheels it might be necessary to readjust the height of the front small wheels as described in **Function description** section. The seat always has to be parallel to the floor.
- For the 24" wheels it is necessary to mount regular wheelchair parking brakes as wheels are not equipped with their own brakes. The brake assembly consists of mounting bracket, Brake Mechanism and 2 fixing bolts.
- Insert the sliding profile of the brake in to the smaller cut out of mounting bracket. Put the mounting bracket on the horizontal side frame tube. Move the sliding profile to a position where the friction pin almost touches the tire (Pic. 4). Tighten the fixing bolts, so the brake is stable and fixed.
- Repeat the assembly process for the other wheel.

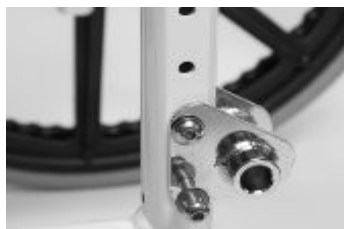Pic. 1: Wheel assembly mounted on the frame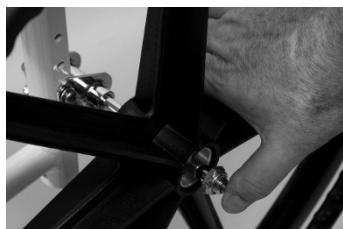

Pic. 2: The 24" wheel with quick release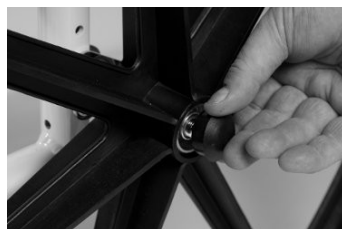

Pic. 3: Cap for quick release button

Pic. 4: Mounting of brakes to the frame

Function description

Adjusting the height of the wheels

To adjust the height of wheels, remove the height adjustment bolt, slide the inner wheel assembly tube to match one of the holes in the frame and fit the bolt through them. There are 3 height settings in steps of 2,5 cm, only the lowest one using the top hole.

WARNING! All 4 wheels should be adjusted to the same height.

Usage of wheel brakes and 24" wheel brakes

To lock the wheel brakes press down on the pedal on the wheel assembly until it locks and repeat for the other wheels. To unlock the pedal brakes, raise the pedal up with the tip of the foot and repeat for the other wheels.

To apply the 24" wheel brakes push the brake levers towards front of the chair. To release brakes pull the lever towards back of the chair.

WARNING! Always engage the brakes when a user is sitting down on or getting up from the chair, leaving the chair with a user in it, or putting the chair away, especially on slopes.

WARNING! The warnings above apply to the 24" wheel brakes! 24" wheel brakes (accessory) must both be engaged when the chair is in a stationary use. They may not be used to brake while driving.

Adjusting the height of the seat

The seat height is dependent on height setting of the wheels. To adjust the seat height, change the height of the wheels.

Adjusting the height of the headrest

To adjust the height of the headrest loosen the bolt on the clamp, adjust to the desired height and tighten it.

Using the armrests

The armrests can be easily lifted up and pushed back down. This feature can help make it easier for the user to get in and out of the wheelchair. Raising the armrests makes it easier to perform personal hygiene while showering.

WARNING! Never leave chair unattended with armrests lifted when there is someone in it.

Footrest functionality

- To adjust the length of the footrest, loosen the footrest adjustment bolt. Move the footrest to the desired length and tighten the bolt. Repeat for the other footrest.
- To remove the footrest, pull the footrest detaching lever. The footrest can be removed when turned away by pulling it upwards. Repeat for the other footrest.
- The footplates can be folded upwards.
- These features can make it easier for the user to get in and out of the wheelchair, while also allowing for easier personal hygiene while showering.

WARNING! Never move the chair with footrests detached when the user is on it. Always put user's feet on foot plates when moving the chair.

WARNING! The length of footrests must be adapted to the height of the user.

NOTE: The length of footrests is correctly adjusted when while sitting the user sits on the wheelchair, the thighs rest flat on the seat and both feet rest on the footplates.

Tilting the chair

To adjust or use the tilt function pull the lever located under the headrest on the backrest, adjust the angle of the chair and release the lever.

WARNING! Never leave patient unattended in the chair with seat tilt adjusted to the maximum front or rear tilt position.

Using the sanitary bucket

The commode wheelchair is designed for people who have difficulty using regular toilets.

To use the bucket of the Commode and shower chair, remove the seat cover and take the bucket out of the chair. Remove the bucket lid and put the bucket back into the chair. After use remove the bucket from the wheelchair again and put the lid back on the bucket. Clean the bucket before another use. Put the bucket back into the chair and put the seat cover back in the seat. If you want to replace the sanitary bucket with a lid, it can be purchased as a spare part (item number: 302101)

Using the device with a standard toilet bowl

To use the toilet with the device, remove the toilet bucket, then move the chair backwards over the toilet bowl and remove the seat cover.

Safety belt and safety bar (accessories)

To attach the safety belt, make sure it has loops at each end, remove the backrest and put the loops around the ends of backrest's tubes. Fasten the loops and make sure the lock of the belt is facing forward and the belt isn't tangled up. Attach the backrest back onto the main frame.

The safety bar is installed to the front of the armrests by removing the caps and removing the screws. Put the assembly tubes of the safety bar into the armrests and secure them with the previously removed screws. The safety bar is removed by pressing the buttons on it's assembly and pulling the bar out, and is inserted in reverse order.

Accessories

The device can be equipped with the following accessories (not a part of the standard equipment):


- ✓ Safety belt
- ✓ 24" rear wheels
- ✓ After-amputation support
- ✓ Security bar
- ✓ Anti-tip
- ✓ Closed seat
- ✓ Seat with round hole
- ✓ Velcro backrest
- ✓ Closed footplate
- ✓ Side support
- ✓ Foam cushion
- ✓ Splash guard

4. Technical data

Commode and shower chair with tilt	Art. 302024	L Art. 302025	24" wheel difference
Seat width	45 cm	50 cm	-
Seat depth	43 cm	43 cm	-
Seat height (adjust.)	55 - 60 cm	55 - 60 cm	-
Seat opening	21,3 x 40,5 cm	21,3 x 40,5 cm	-
Pass over height *	40 - 45 cm	40 - 45 cm	-
Pass over width	35 cm	40 cm	-
Seat/Back angle	90 °/ 12°		-
Distance between armrests	46,5 cm	51,5 cm	-
Backrest height	46 cm		-
Total width	56 cm	61 cm	+ 14 cm
Total depth	110 - 120 cm	110 - 120 cm	+ 19 cm
Total height (adjust.)	97 - 141 cm	97 - 141 cm	-
Max. user weight	150 kg	150 kg	-
Weight	15,10 kg	15,70 kg	+ 4,20 kg
Tilt angle	-5° to +25°	-5° to +25°	-

* +5 cm with the new accessory 30212M

Product label

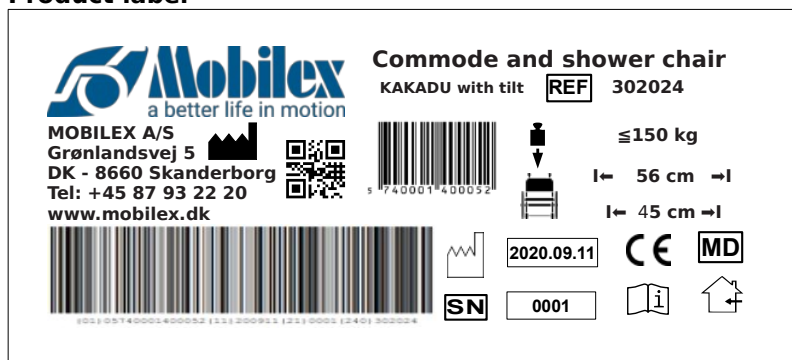Example of the product label

	QR-code link to homepage		Manufacturing date
	Manufacturer		Loading force
	Observe the user manual		Max width
	Only to be used indoor		serial no
	CE marked product		Item Number
	Medical Device		

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. The entire product can be disinfected with a commercially available disinfectant. Always follow the disinfectant manufacturer's instructions for use and safety when disinfecting.

Wheels can be cleaned with hot water with a mild cleaning agent. After cleaning the wheel bearings should be lubricated with oil periodically or when the wheels start squeaking.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

Cleaning the bucket

The bucket must be emptied and disinfected after use. Do not use water with a temperature higher than 80°C.

When using a steam cleaner/autoclave: Remove the handle from the bucket and clean the bucket after it has been laid flat down. Do not use the steam cleaner/autoclave on the bucket for more than 10 minutes. Do not place the bucket at an angle to avoid deformation during cleaning.

WARNING! If the product is exposed to very calcareous water, a binding and accumulation of lime on wheel bearings and other moving parts may occur. In this case, the product can be cleaned with a limescale remover and then lubricated.

WARNING! When used in highly chlorinated environments extra strong corrosion may occur and the warranty will be void.

WARNING! If the frame is bent or broken, the product must not be regenerated but must be replaced.

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

It is recommended to disassemble the device for transportation. The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

If the device needs to be carried without being fully disassembled, it is recommended to remove the footrests and lift the device by grabbing it by the seat tubes on the sides.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

<i>Problems</i>	<i>Possible causes</i>	<i>Solutions</i>
The chair is not stable and can't reach horizontal position	The wheels are adjusted to different heights	Adjust all wheels to the same height
The wheels do not turn smoothly and easily	The space between wheels and forks is filled up with dirt	Clean the space between wheels and forks of all unintended objects and elements

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

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