RAISED TOILET SEAT

Art. 301101, 301102, 301103, 301104, 301105, 301106

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

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mobilex.dk/medias/2021-07/ce_toiletsaedeforhoejere_301101_06_8284.pdf

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

The toilet seat raiser is a medical device which is intended to be used by elder or disabled persons only as a seating option and additional support during usage of a toilet. Any other use is prohibited.

Indications

Raised toilet seats are overlays for regular toilet bowls, indicated for use as a seating device by people with functional limitations, persons in wheelchairs and other people with limited mobility of the lumbar spine, knees, or hips, and for all those who have difficulties with seating down low and standing back up from toilet's regular height. Using the seat for purposes other than those indicated above, is prohibited.

Contraindications

Raised toilet seat is not indicated for those patients who can't maintain proper body balance in a sitting and standing position. The seat must not be used if the user has problems with impaired perception. Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S	Phone:	+45 87 93 22 20
Grønlandsvej 5	Telefax:	+45 87 93 17 77
DK-8660 Skanderborg	Email:	info@mobilex.dk
Denmark		

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- Do not sit on the lid,
- Make sure the device is securely attached before use.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Raised toilet seat;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

The Toilet Seat Raiser is a stable seating support that allows for easier sitting and getting up by raising the height of the seat.

Assembly and use description

Loosen the plastic knobs on both sides of the device. Lift the existing toilet lid and toilet seat, and put the device directly on the toilet bowl so it lies flat and stable. The inner bottom ring should be located inside the toilet hole. Tighten both knobs so the device is locked in position. Ensure that the device is locked and stable as well as cleaned (See **chapter 5**) before each use.

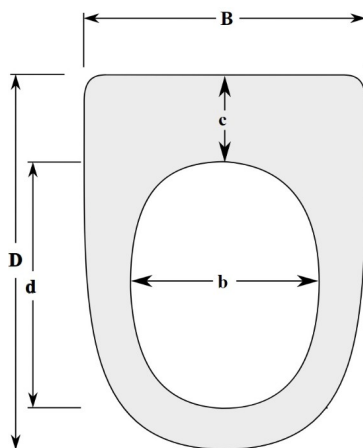
Accessories

There are no accessories provided for this device.

4. Technical data

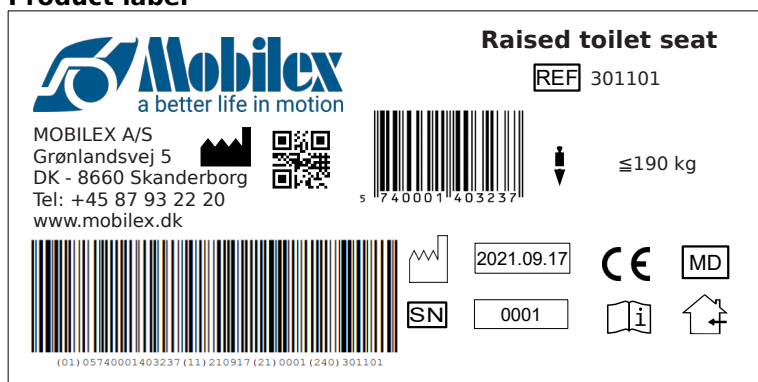
Raised toilet seat	Total width	Total depth	Total height	Seat width	Seat depth	Seat height	Max. user weight	Material
Art. 301101+02	36 cm	40 cm	10 cm	22 cm	27 cm	5 cm	190 kg	PP
Art. 301103+04						10 cm		
Art. 301105+06						15 cm		

Dim.	301101-06
D	40 cm
d	27 cm
B	36 cm
b	22 cm
c	5 cm
G*	31 - 38,5 cm

* G is the internal distance between the jaws of the fastening clamps

Product label

Example of the product label

	QR-code link to homepage		Manufacturer
	CE marked product		Item Number
	Observe the user manual		Max load
	Medical Device		Manufacturing date
	To be used indoor		Serial no

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. For disinfection, use only disinfectants certified for use with medical devices available locally.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the product for bends, cracks, damage, wear or corrosion
- ✓ General re-tightening of the components

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

Problems	Solutions
The device isn't stable and moves	Check the fixing of the device to a toilet, tighten the plastic nuts if necessary.

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at

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