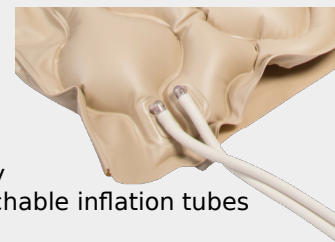
PRESSURE RELIEVING MATTRESS B01+P05

Art. 270016

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

Alternating cycle bubble pad.

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mobilex.dk/medias/2021-07/trykaflastningsmadras_270016_270017_7948.pdf

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

This Mobilex product is a medical device which is intended to be used only as an air support surface system for the human body in laying position. It is suitable for pressure ulcer prevention.

Indications

The device has been specifically designed for prevention of bedsores and pressure ulcers and offers an affordable solution to 24-hour pressure area care. The mattress pad can maximally be loaded with 130 kg.

Contraindications

Pressure relief therapy is not suitable for patients with cervical or sceltal traction or with unstable spinal cord injuries.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
---	--

2. Safety requirements

General safety

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately **stop** using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't stand on the mattress pad.
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must only be used on beds equipped with regular mattresses. It is forbidden to put the anti-decubitus mattress directly on the bed frame.
- The device is dedicated to be used indoors only.
- A physical/occupational therapist should always assist in first adjustment of the device for optimum support and correct functionality.
- The pressure relieving mattress should always be covered with a sheet so that the patient never lies directly on the pressure relieving mattress with the skin in contact with the mattress.

Electrical safety

- To avoid incidents and to reduce the risk of burns, electrocution, fire, or injury to persons while using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:
- Always unplug this product immediately after use.
 - Do not use while bathing or near water sources.
 - Do not place or store product where it can fall or be pulled into a tub or sink.
 - Do not place in or drop into water or other liquids.
 - Do not reach for a product that has fallen into water. Unplug immediately.
 - Close supervision is necessary when this product is used by, on, or near children or disabled people.
 - Use this product only for its intended use as described in this user manual.
 - Do not use attachments not recommended by the manufacturer.
 - Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the product to a service center for examination and repair.
 - Keep the cord away from heated surfaces.
 - Never block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and the like.
 - Never drop or insert any object into any opening or hose.
 - Do not use outdoors or operate where aerosol (spray) products are being used.
 - Before any connection is made, make certain that this product is connected safely to a conductor via the two-wire power cord.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Pump;
- ✓ 1 Pressure relieving mattress;
- ✓ 1 Inflation tube package;
- ✓ 1 Repair kit;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

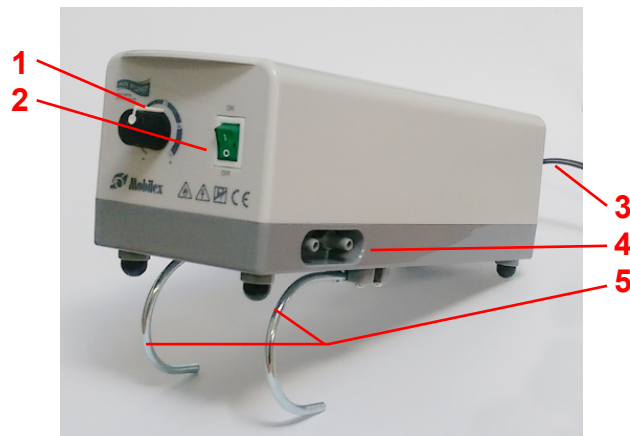
The device comes with all the necessary features for efficient decubitus prevention. It consists of few various elements. No tools are required for assembly.

The device consists of the following elements:

1. Pressure-adjustment knob
2. Power Switch
3. Power Cord
4. Inflation nozzles (for inflation tubes)
5. Foldable hook hangers

Assembly description

- Lay the pressure relieving mattress on a foam base or other regular bed mattress. Do not put it directly on the bed frame.
- The inflation inlets (A) should be towards the foot end so that they can be connected to the inflation nozzles on the pump.
- The pressure relieving mattress **must** be covered with a cotton sheet to avoid direct skin contact and reduce friction.
- Hang the pump over the bed frame or board at the foot end of the bed using the foldable hook hangers. Make sure the pump is secured.
- Connect the ends of the inflation tubes to the pressure relieving mattress inlets and to the pump's inflating nozzles. Make sure the tubes are properly attached and that they are fitted tight on the nozzles.
- Make sure the inflation tubes are not kinked or tucked under the pad.
- Before inserting the plug into the outlet, make sure the voltage is compatible.
- Plug the power cord into an electrical outlet with AC power. Replace or repair if damaged.
- Make sure to disconnect the pump by unplugging the power cord when it is not in use.

Function description

Power button and first inflation

Turn on the power by switching the power switch at the right side of the pump. The indicator of the power switch will come on. The pump will start to pump air into the mattress pad. For time-saving in the first inflation, turn the pressure knob to the full Firm/Hard position (+). It requires approximately 10 minutes for a bubble pad to be fully inflated.

Pressure adjustment

When mattress pad is fully inflated turn the pressure-adjust knob to set a comfortable pressure level from soft (-) to firm (+) between settings 1 to 5 according to patient's weight and comfort. The pump pressure is correctly set, when the patient's body does not rest on the underlying mattress. Users can then easily adjust the pressure level of the bubble pad to a desired firmness by themselves or according to the suggestion from a health care professional. If the pressure is consistently low, check for any leakage occurring in inflation tubes. Contact your local dealer for replacement of any damaged tubes.

NOTE: The deflating/inflating cycle runs automatically, when pump is turned on.

Accessories

There are no accessories provided for this device.

4. Technical data

Pressure relieving mattress B01+P05	Pump	Mattress Pad
Dimensions	27,5 x 10 x 10 cm	200 x 90 x 6 cm
Material	-	PVC 0.3 mm
Min./Max. Load	-	20 / 130 kg
Weight	1.1 kg	-
Pressure Range	70 ~ 130 mm Hg	-
Cycle Time	6 min	-

System	Specification
Classification	Class I
Applied Part	Mattress Pad
Input Rating	230V 50Hz, < 10W, 1A
Fuse Rating	250V, F1AL
Air Output	6-7 L/min
Environment Requirements	<ul style="list-style-type: none"> • Temperature: Operation 0°C to 40°C • Humidity: Operation 30%~75% non-condensing
Safety Standards	CE, Protection Against Class I Electric Shocks

Product label

Example of the product label

	QR-code link to homepage		Manufacturer
	CE marked product		Item Number
	Observe the user manual		Max load
	Medical Device		Manufacturing date
	To be used indoor		Serial no

Meaning of the icons

5. Care and maintenance

It is important to follow these procedures for cleaning and decontamination of the pump and bubble pad before using the system on patients. The cleaning is required at least once a week to maintain personal hygiene.

Pump

- DO NOT immerse or soak the pump.
- Check for external damage and move the pump to a cleaning area.
- Place the pump on a work surface and spray or wipe the outside of the case with quaternary ammonium solution.
- DO NOT spray any cleaning solution directly on the surface of the pump.
- DO NOT use a Hypocarbonate or Phenolic based cleaning solution as this may cause damage to the case. Allow the solution to incubate for 10 minutes or accordingly as stated by the cleaning product used.
- Spray cloth with cleaning solution and clean faceplate. DO NOT allow excess cleaning solution on faceplate or control panel (if solution gets inside, damage will occur).
- Allow surface to thoroughly dry after cleaning.
- After the pump is thoroughly cleaned and dried, proceed to plug in the pump and test to see if it runs normally.

Pressure relieving mattress

- Any obvious blood spots should be wet thoroughly with 1:9 Hypochlorite solution (1 part bleach to 9 parts water) and allow drying for at least 10 minutes. Then use a clean, damp cloth.
- Brush or wipe down all surfaces with soap and water before applying any liquid.
- Allow it to thoroughly air dry. When the underside is dry, turn the pressure relief mattress over: Then wipe the top side of the pressure relief mattress with disinfectant..
- Dry the pressure relieving mattress on a sunless area after cleaning.

Repair kit

- If the position of the air leakage is determined, clean a big enough area for the patch around it and put a layer of adhesive on the clean area. Make sure the adhesive doesn't get into the hole.
- Put the patch on the adhesive, apply pressure with a heavier object and give it time to cure.

Maintenance

- Check the power cord and plug to see if there are abrasions or excessive wear.
- Plug in the pump and check the airflow from the hose connection port. The airflow should alternate between ports every half-cycle time.
- For storage lay the pressure relieving mattress flatly. Fold or roll it carefully. Unplug the pump and store with proper identification tag. Store device in dry places with non freezing temperatures.
- Check the pressure relieving mattress for any air leakages through punctures or welds.

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

<i>Problems</i>	<i>Solutions</i>
The pump doesn't work	<ol style="list-style-type: none">1. Check if the plug is inserted firmly into the outlet.2. Turn on the power switch again.3. If the power indicator is ON and the pump still doesn't work, contact your local dealer immediately.4. If the power indicator is OFF, there may be a faulty outlet. Try to connect the power cord to another outlet. If the power indicator is still OFF, contact a qualified electrician for main power check.
Incomplete inflation (Low pressure)	<ol style="list-style-type: none">1. For a quick check, adjust the pressure to Firm.2. Check to see if the inflation tubes connected to the pump are twisted or disconnected, or any leakage occurring. Always keep the tubes straight. Change the tubes if there is any leakage.3. Check the pressure relieving mattress for any air leakages.

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

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