- (GB)** **Standing swivel plate - user manual**
- (D)** **Steh-Drehscheibe - Gebrauchsanleitung**
- (DK)** **Drejeskive, stående - brugervejledning**
- (I)** **Disco girevole da pavimento - Manuale d'uso**

*Item no.*  
*Art.Nr.*  
*Varenr.*  
*Cod.*

**278058**

**MOBILEX A/S**  
Grønlandsvej 5  
DK-8660 Skanderborg  
www.mobilex.dk

**GB Standing swivel plate**

**Purpose**

The product is an aid designed to help people with movement difficulties. Product can be used indoor or outdoor. Can be used to turn a person standing. Use the product only for the maximum allowable weight as indicated on the label. If you have further questions, please contact your dealer or Mobilex A/S directly.

**Safety instructions**

1. Use the product only in perfect condition and only for the intended purpose.
2. If the product shows damage or defects, please contact your dealer immediately.
3. Read the information on the label.
4. Do not leave the product unattended on the floor- someone could step on it, slip and get seriously injured!
5. Do not make any changes to the product except with the written permission of the manufacturer.

**Product description**

The swivel plate fits surfaces and allows the user to turn easy when standing on the swivel. The product can help people with back or knee problems to avoid unnecessary rotation of the back. The product can be used when a person needs to be moved from one place to another. The product is made of high quality polyester and low-friction material that provides good comfort and durability through a silicone anti-slip technology.

**Use of the product**

1. Never use the swivel plate without help from a helper.
2. Place the cushion on the floor or ground.
3. Person that need to be turned place both feet on the swivel plate and the helper support the person while using the swivel plate.
4. Person can now stand up, never stand up unsupported on the swivel plate.
5. The helper turns now the person to the new position
6. Person can now sit down, with help from the helper in the new position

**Maintenance and care**

Use a soft cloth only with standard, neutral cleaning agents. Do not use solvents or chlorine!

**Recommended service life:** Normally 3 years depending on use.

**Warning**

 **Please note that if the product is bent or broken, the product must not be recycled but must be replaced.**

**Reuse**

The product is suitable for recycling. Before recycling, where a new user is to take over the product, the product must be checked according to above mentioned advice.

**Disposal of the product**

The product can not be disposed of with household waste but must be brought to the local recycling centre.

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Example of the product label

## Warranty terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

## Declaration of Conformity (CE)

(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,

### hereby declares that the

(erklært hiermit, dass das Produkt / erklærer hermed, at / dichiara che il)

### Standing swivel plate

Item no. 278058

### is designed and produced in accordance with the technical documentation and the basic requirements of the:

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### In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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