**ARTHRITIS ROLLATOR TIGER**  
"OUTDOOR"  
Art. 312205, 312206

**Gebrauchsanleitung**  
**Brugervejledning**  
**User Manual**  
**Manuale**  
**Bruksanvisning**  
**Instrukcja Obsługi**  
**Manuel de l'utilisateur**

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## 1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

### Intended use of the device

This product is medical device which has been designed to increase the mobility of persons who have difficulties standing or walking. The rollator enables those persons to regain the ability to perform daily activities. The rollator is suitable for persons who as a result of various diseases need stable walking support. The rollator should only be used as a support for standing and walking. Any other use is prohibited.

### Indications

The rollator is a medical device designed to help people with functional limitations. The indications for use are diseases that cause mobility limitations and disabilities, obstructing independent sitting, standing and walking. Therefore, there is a need for an additional stable support when performing the activities listed above. Indications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status by professional physician or physical therapist, but main target user groups are elderly people, persons with paresis, rheumatic diseases, and other people with limited mobility of the lumbar spine, knees or hips. Using the product for any purpose other than those described above is prohibited.

### Contraindications

Rollators are not indicated for those persons who can't maintain proper body balance in a standing position. The device must not be used if the user has problems with perception. Contraindications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter

### Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

### Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S	Phone:	+45 87 93 22 20
Grønlandsvej 5	Telefax:	+45 87 93 17 77
DK-8660 Skanderborg	Email:	info@mobilex.dk
Denmark		

### Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device and the bag must not be loaded with more weight than indicated in Technical data.
- All wheels MUST be in contact with the floor at ALL TIMES during use. This will ensure that the rollator is properly balanced and should decrease the risk of accidents.
- Use caution when assembling or adjusting any parts of the device.
- Always check if the brakes function properly before use.
- Use the rollator only when it has been fully unfolded and the locking device has been activated.
- The seat may only be used when parking brakes have been activated.
- Both brakes should always be activated or locked simultaneously.
- It is not allowed to carry a person or object on the seat while the rollator is in motion.
- Parking brakes must always be engaged when the rollator is not in use.

### Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Rollator folded incl. fitted seat, backbelt and shopping bag;
- ✓ 1 Crutch holder (found in the shopping bag);
- ✓ 1 User manual;

### Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

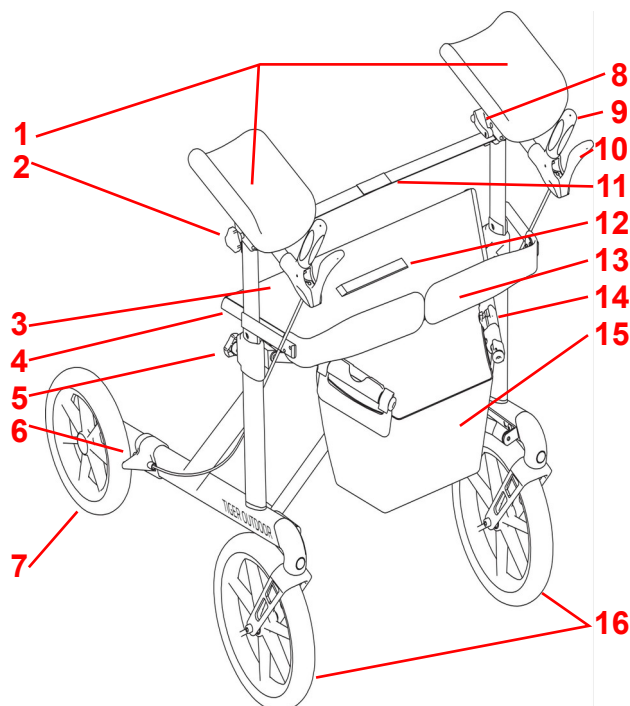
### 3. Description

#### Product description

The rollator has a powder coated frame and 4 wheels. The light-weight frame makes it easy to fold it together and offers easy storage and transport. The rollator runs easily and is sufficiently stable. The height of the underarm supports can be adjusted. The rollator is delivered with a mounted seat, a backbelt and a shopping bag.

#### The rollator consists of the following elements:

1. Underarm supports
2. Brake-handle depth adjustment star-knob
3. Seat
4. Seat tube
5. Underarm support height adjustment star-knob
6. Brake
7. Rear wheel
8. Handle adjustment lever
9. Push handle
10. Brake lever
11. Frame stabilizing strap
12. Folding strap
13. Backbelt
14. Folding lock button
15. Shopping bag
16. Front turning wheel

#### Assembly description

To assemble the rollator, attach the two forearm supports by removing the frame stabilizing strap, inserting the forearm supports into the grey tubes from the top and re-attaching the frame stabilizing strap to the forearm supports with the two screws. Attach the two handles by inserting the push handle tube into the forearm supports and tightening the handle depth adjustment star-knob.

#### Function description

##### Folding/unfolding the frame

To unfold the frame press the folding lock button and press down on the seat tubes until the frame unfolds and "clicks", which indicates the rollator is locked in an unfolded position. To fold the frame pull up the folding strap and finish folding the frame by pushing seat tubes together until the folding lock button "clicks".

**WARNING!** Seat tubes must snap into place firmly on both sides when unfolding the rollator. All parts must be firmly locked before the rollator can be used. Failure to observe these warnings may result in injury.

##### Underarm support height adjustment

To adjust the height of the underarm support, first detach the frame stabilizing strap, loosen the height adjustment star-knob and slide the support to a desired setting. Tighten the star-knob and repeat for the other support. After adjusting both supports, reattach the frame stabilizing strap.

**NOTE:** Both underarm supports should always be at the same height.

##### Underarm support rotation adjustment

To adjust the horizontal rotation of an underarm support loosen the star-knob **1** under the underarm support, adjust the rotation and re-tighten the star-knob **1**. Repeat for the other support. Adjust the stabilizing strap if it becomes loose or too short.

##### Push handle depth and tilt adjustment

Loosen the lever **2**, slide the tube of the handle to adjust the depth or rotate it to tilt the handle, and re-tighten the lever **2**. Adjust the stabilizing strap if it becomes loose or too short.

##### Brakes

The brakes can be gradually activated by pulling the brake levers back with increasing force. Brakes are fully engaged when brake levers touch the handles and the rollator is fully stopped. To stop braking let go of the brake levers.

The rollator is also equipped with a parking brake function. To engage the parking brake push the brake levers forward. To disengage the parking brake, pull the brake levers back to a neutral position.

**WARNING!** Always activate the parking brake when not using the rollator! When placing the rollator on a non-plane surface you must always activate the parking brake!

**WARNING!** The brakes must always be in a locked position when the rollator is parked! Never use the seat if the brakes are not locked! When seated, never lean backwards. It is recommended to sit only at the front part of the seat. Do not sit down on the bag.

**WARNING!** Before sitting on the rollator the frame stabilizing strap must be released. When the rollator is used to walk with, the frame stabilizing strap and the backstrap must always be correctly mounted on the rollator.

#### Accessories

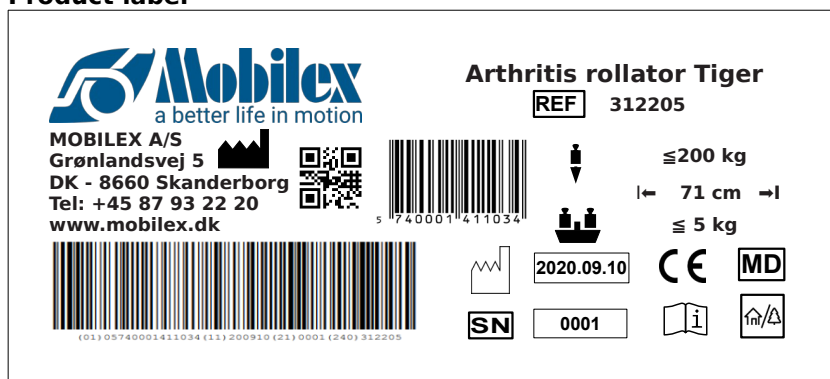
The device can be equipped with the following accessories (not a part of the standard equipment):

- ✓ Fixed seat
- ✓ Speed reducing device
- ✓ Closed bag with a zipper
- ✓ Netbag with a zipper
- ✓ Red/blue bell
- ✓ LED lights White/Red
- ✓ Oxygen Bottle holder
- ✓ Tray

#### 4. Technical data

Arthritis Rollator Tiger	Art. 312205	Art. 312206
Colors	antracite grey	metallic brown
Height of push handles (adjustable)	87 - 110 cm	
Recommended user heights	140 - 185 cm	
Seat height	62 cm	
Total width unfolded / folded	71 / 31 cm	
Seat width / Inner width between push handles	56 / 46cm	
Length	77 cm	
Weight	12 kg	
Max. user weight / bag load	200 kg / 5 kg	
Brakes	2 lockable with parking function	
Wheel sizes (front and back)	200 x 35 mm	

#### Product label

Example of the product label

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	Only to be used indoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device

Meaning of the icons

#### 5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. For disinfection, use only disinfectants certified for use with medical devices available locally.

Wheels can be cleaned with hot water with a mild cleaning agent. After cleaning the wheel bearings should be lubricated with oil periodically or when the wheels start squeaking.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

**Recommended service life:** Normally 10 years, depending on use.

**Storage:** We recommend storage in a dry place in temperatures above 0 °C.

#### Transportation

It is recommended to fold the device for transportation. The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

#### Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

## 6. Most common problems and solutions

<i>Problems</i>	<i>Possible causes</i>	<i>Solutions</i>
The rollator is difficult to guide or push	Dirt between wheels and forks	Remove dirt or hair between wheels and forks
The brakes do not work or work only on one side	The brake shoes are worn, or the brakes are not adjusted correctly	Adjust the brakes. Replace the brake shoes if needed (→ service center)
The seat does not fit the frame while unfolding the rollator	The locking device is in a wrong position or defective	Check that the locking device is in a correct position. Replace it if needed (→ service center)
The backbelt or the frame stabilizing strap cannot be fastened	The backbelt or the frame stabilizing strap weren't fastened BEFORE the rollator was unfolded.	Fasten the backbelt or the frame stabilizing strap BEFORE unfolding the rollator.

## 7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

## 8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

## 9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at

[mobilex.dk/medias/2021-08/ce\\_arthritis\\_rollator\\_tiger\\_outdoor\\_312205\\_6\\_8420.pdf](http://mobilex.dk/medias/2021-08/ce_arthritis_rollator_tiger_outdoor_312205_6_8420.pdf)

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