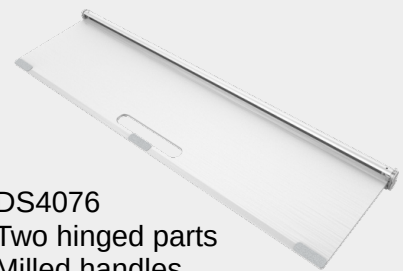
Doorstep ramps

Art. DS2176, DS4025, DS4076

Montage- und Gebrauchsanleitung
Montage- og brugervejledning
User Manual
Manuale d'uso e manutenzione

Doorstep ramps
Mobile aluminium doorway ramps

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Indholdsfortegnelse

Table of content

Indice

	Montage- und Gebrauchsanleitung	3- 5	DEUTSCH
	Montage- og brugervejledning	6-8	DANSK
	User Manual	9-11	ENGLISH
	Manuale d'uso e manutenzione	12-14	ITALIANO
	Declaration of Conformity	15	Doc

1. Einleitung

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1. Introduction

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

Indications

Ramps are indicated for people with mobility limitations who due to problems with walking, standing or sitting are forced to use mobility devices such as wheelchairs, scooters, rollators etc. Ramps are indicated for elimination of architectural obstacles by smooth connection of the level differences that a wheelchair user has to overcome and thus they allow the wheelchair user to reach places normally inaccessible to him/her.

The ramp is intended for use independently or with the help of an attendant. Using the ramp for purposes other than those indicated above, is prohibited.

Contraindications

Contraindications cannot be tied to one, specific clinical picture. The choice of a ramp type and length that is appropriate for a given must be always made with the help of a specialist physician or physiotherapist.

By questions please take contact to your dealer or directly to Mobilex A/S.

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 E-Mail: info@mobilex.dk
--	---

2. Safety requirements

- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Be aware of the information on the product label.
- Make sure that this user manual is read by all persons using the ramp. Damages, caused by the fact that the user manual has not been followed, will not be accepted.
- Ramps must not be loaded with more than the "max load" described in Technical Data and on the label.
- The ramps can be used at maximum inclination as described in Technical Data and on the label.
- Only use the ramp for the purpose it is intended for.
- Avoid making constructive changes on the ramp, unless you have the producers written accept.
- The ramp must only be used on a stable surface.
- Be careful during folding or unfolding of the ramp. There is a risk of finger squeeze or cut between parts of the ramp.
- It is important that the legs of the DS2176 ramp are adjusted so that all 3 legs are supported on the ground so that the ramp does not tilt.
- On ramps DS4025 and DS4076 it is important that the middle of the ramp (the round profile on the back-side of ramp) has support (between the back of the ramp and the ground), otherwise the ramp may tilt up.

3. Description

Product description

The DS ramps are delivered in a box. The model DS4025 also comes with a carrying bag and the model DS2176 with 2 x 3 adjustment screws

The contents are:

- ✓ Ramp
- ✓ 1 User manual

Assembly and usage

DS4025 / DS4076: The ramp consists of two hinged parts and is delivered assembled. The ramp model DS4076 consists of a single wide part, while model DS4025 consists of two separate narrower parts. To use, place the ramp over the obstacle to be crossed (max. height 5 cm) so that the two sides with the shaped handles are on opposite sides of the obstacle.

DS2176: The ramp consists of a single part that can be adjusted in height with the 2 x 3 adjustment screws (A + B) provided. Use the 3 shorter screws for heights of 3.5-4 cm and the longer screws for heights of 4 to 7 cm.

Transport recommendations

The foldable DS4025 comes with a bag that makes the ramp easy to transport. The DS4076 is of the same type and can be folded and transported using the milled carrying handles. The DS2176 can be transported as required although this ramp is intended as a permanent solution on site.

Accessories

There are no accessories provided for these ramps.

4. Technical data

Technical data	DS2176	DS4025	DS4076
Size	21 x 76 cm	40 x 25 cm	40 x 76 cm
Crossable height	3,5 - 7 cm	2 - 5 cm	2 - 5 cm
Weight	1,55 kg	2 kg (pair)	3,3 kg
Max. load	350 kg	350 kg	350 kg

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Batch no/ serial no
	CE-marked product		Item Number
	To be used indoor and outdoor		Medical Device
			Max Elevation / height

Overview icon meaning

Product label

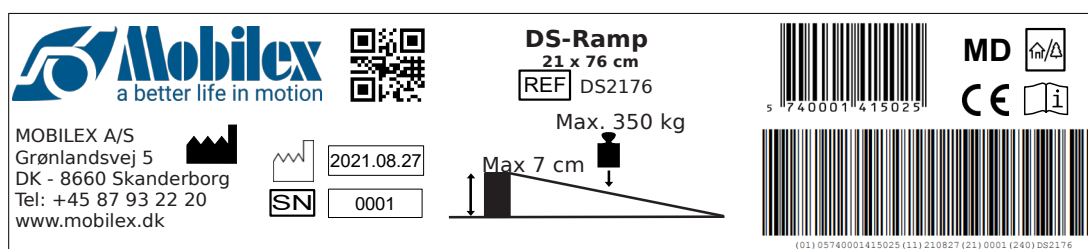
Image of the product label for DS2176 ramp

5. Maintenance and cleaning

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional

- Lighter dirt and dust can be removed with a wet cloth.
- Heavier types of dirt can be removed with a non-corroding, mild washing agent.
- You can also use garden hose to clean the ramp.

Recommended service life: Normally 10 years depending on use.

Warning

 Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

Storage

We recommend storage in a dry place.

Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

6. Most common problems and solutions

<i>Symptoms</i>	<i>possible cause</i>	<i>what to do?</i>
Ramp is not stable	The surface under the ramp is not flat	Check the position of the ramp. Put it on flat, smooth surface
Ramp is not stable	The legs under the ramp do not support evenly	Adjust so that all 3 legs are of the same length and adjusted to the same height

7. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices. The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

PRODUCT LIST

Doorstep ramps:

REF / item no.	DS4025	DS4076	DS2176
UDI-DI	5740001415018	5740001415001	5740001415025
BASIC-UDI-DI	57400014DOORSTEPRAMPSZM		

Harmonized norms used during conformity estimation:

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In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark, **SRN: DK-MF-000021885**