## SC ramps

Item no. SC-045, SC-060, SC-090, SC-120, SC-150 & SC-180

**Montage- und Gebrauchsanleitung**  
**Montage- og brugervejledning**  
**User Manual**  
**Manuale d'uso e manutenzione**

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## 1. Introduction

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

### Intended use of the device

The Mobilex SC ramps are medical devices which is intended to be used only to assist patients in wheelchairs or scooters. They enable the passage of elevated points (e.g. Stairs) with or without the assistance of a helper. Mobilex Ramps thereby enables patients, sitting in a wheelchair because of invalidity, to enter or leave buildings or cars, buses etc. The ramps can be used as well inside as outside. Any other use is prohibited.

### Indications

Ramps are indicated for people with mobility limitations who due to problems with walking, standing or sitting are forced to use mobility devices such as wheelchairs, scooters, rollators etc. Ramps are indicated for elimination of architectural obstacles by smooth connection of the level differences that a wheelchair user has to overcome and thus they allow the wheelchair user to reach places normally inaccessible to him/her. The ramp is intended for use independently or with the help of an attendant. Using the ramp for purposes other than those indicated above, is prohibited.

### Contraindications

Contraindications cannot be tied to one, specific clinical picture. The choice of a ramp type and length that is appropriate for a given must be always made with the help of a specialist physician or physiotherapist. Observe the safety information in chapter 2.

By questions please take contact to your dealer or directly to Mobilex A/S.

<b>Mobilex A/S</b> Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 E-Mail: info@mobilex.dk
--	---

## 2. Safety requirements

- Make sure that this user manual is read by all persons using the device. Manufacturer doesn't take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- During different adjustments which are possible on the device the user must take care not to squeeze the fingers or other body parts between two moving elements of the device
- The device must only be used on a stable surface.
- Ramps must not be loaded with more than the "max load" described in Technical Data and on the label.
- The ramps can with assistant be used at maximum inclination of:

<b>SC-045:</b>	<b>9 cm</b>
<b>SC-060:</b>	<b>12 cm</b>
<b>SC-090 :</b>	<b>18 cm</b>
<b>SC-120 :</b>	<b>24 cm</b>
<b>SC-150:</b>	<b>30 cm</b>
<b>SC-180:</b>	<b>36 cm</b>

- Be careful during folding or unfolding of the ramp. There is a risk of finger stuck or cut between parts of the ramp.

### 3. Description

#### Product description

Mobilex SC-Ramps are foldable platforms made of aluminium. The surface is specially “rib” shaped for anti-slipping purposes.

SC-Ramps are supplied in cartons. The contents are:

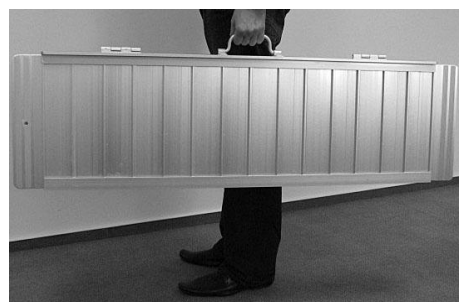
- ✓ 1 Ramp
- ✓ 1 User manual

#### Transport recommendations

Mobilex SC ramps can be folded for transport and locked. Always assure that ramp during transportation is safely secured against unintended movements.

#### Accessories

For SC ramp SC-060 a bag for transporting the ramp is available as accessory with item no. SB-060. Contact your dealer for more information.

### 4. Technical data

SC Ramps	SC-045	SC-090	SC-090	SC-120	SC-150	SC-180
Length	46 cm	62 cm	92 cm	122 cm	152 cm	182 cm
Width	77 cm	77 cm	77 cm	77 cm	77 cm	77 cm
Surface width	76 cm	76 cm	76 cm	76 cm	76 cm	76 cm
Height of side protect	3,5 cm	3,5 cm	3,5 cm	3,5 cm	3,5 cm	3,5 cm
Max. elevation / height	12° / 9 cm	12° / 12 cm	12° / 18 cm	12° / 24 cm	12° / 30 cm	12° / 36 cm
Weight	4,50 kg	6,50 kg	9,00 kg	12,00 kg	15,00 kg	18,00 kg
Max. load	300 kg	300 kg	300 kg	300 kg	300 kg	300 kg
Folded size cm (LxWxH)	45 x 40 x 8	62 x 40 x 8	92 x 40 x 8	122 x 40 x 8	152 x 40 x 8	182 x 40 x 8

### Product label

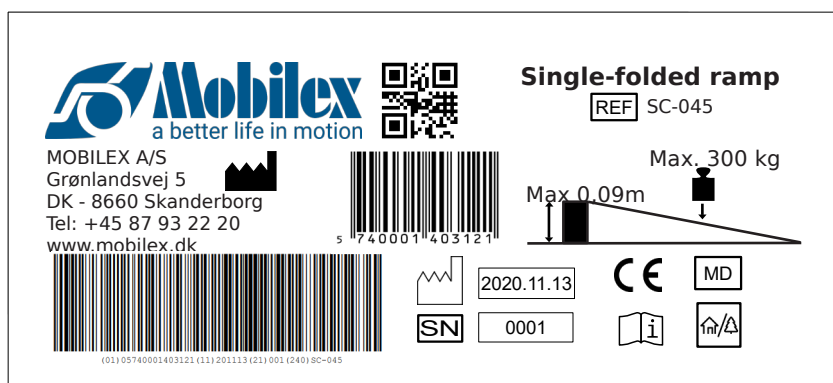
Image of the product label (SC-045)

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Batch no/ serial no
	CE-marked product		Item Number
	To be used indoor and outdoor		Medical Device
	Max Elevation / height		

Overview icon meaning

### 5. Maintenance and cleaning

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional

- Lighter dirt and dust can be removed with a wet cloth.
- Heavier types of dirt can be removed with a non-corroding, mild washing agent.
- You can also use garden hose to clean the ramp.

**Recommended service life:** Normally 10 years depending on use.

**Warning**

 Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

**6. Reuse**

The product is suitable for recycling. Before recycling, where a new user is to take over the product, the product must be checked according to point 5.

**7. Most common problems and solutions**

<i>Symptoms</i>	<i>Possible cause</i>	<i>What to do?</i>
The ramp jams when folded and when unfolded	the side wheel protectors could be bent	try to straighten the wheel protectors
The ramp is difficult to fold or unfold.	Dirt on or inside the hinges	Clean and oil the hinges

**8. Disposal of the product**

The product can not be disposed of with household waste but must be brought to the local recycling centre.

**9. Guarantee terms**

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,

**Hereby declare under our sole responsibility as a legal manufacturer** that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

**PRODUCT LIST**

**Single-folded ramp**

Item number: **SC-045, SC-060, SC-090, SC-120, SC-150, SC-180**

**ACCESSORIES LIST**

Item no.	Accessories item no.
SC-060	SB-060

Harmonized norms used during conformity estimation:

EN 12182:2012, PN-EN ISO 14971:2012, EN 1041:2009

**In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /**

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