Shower board Item No. 302034

Montage- und Gebrauchsanleitung
Montage- og brugervejledning
User Manual
Manuale d'uso e manutenzione

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1. Introduction

Thank you for buying a high-quality Mobilex product. This user manual contains a description of the medical device and important guidance to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

Intended use

The shower board is a medical device which intended use is to give stable & easy access and seating support when disabled persons use a bathtub. It is capable of being adjusted to suit the majority of bathtub widths.

Indications

Bath benches and seats are overlays for regular bathtubs, indicated for use as a seating device by elderly people, persons in wheelchairs and other people with limited mobility of the lumbar spine, knees, or hips, and for all those who have difficulties with seating down low and standing back up from bathtub's bottom while performing everyday personal hygiene. Using the seat for purposes other than those indicated above, is prohibited.

Contraindications

This product is not indicated for those patients who can't maintain proper body balance in a sitting and standing position. The device must not be used if the user has problems with impaired perception. Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

If you have any questions, please contact your dealer or Mobilex A/S.

| | |
|--|--|
| Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark | Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk |
|--|--|

2. Safety requirements

- Only use the product for the purpose it is intended for
- Do not stand on the bath board
- Only use the product when it is in technically perfect condition
- Avoid overloading the product
- Use only on flat, smooth walls in the bath
- The bath board must be precisely adapted to the bathtub width
- Always check the stability of the product before use
- The bath board must be mounted so that it cannot move in the bath
- The surface of the bath must be well supported by the sides of the bath

3. Description

Product description

Supplied parts and assembly instructions

The bath board is made of recyclable white plastic. The smooth surface makes it easy to clean. The bath board is easy to assemble. The easily adjustable, robust securing plugs optimally support the board on the inside of the bathtub. The handle supplied as standard is easy to grip and increases safety.

Parts supplied:

- ✓ 1 bath board
- ✓ 1 handle
- ✓ 4 legs
- ✓ This instruction manual

Assembly

The handle must be mounted in the holes using the 2 screws provided. The screws are fitted from the underside of the board up into the handle. This can be done with a screwdriver or a coin. The handle is then mounted on the bath board. Now mount the legs. This is done by unscrewing the screw from the leg. The screw is mounted from the top of the board and the leg from the bottom, the leg can be tightened with the hands.

Warning! Before using the board, it is important that the board is mounted in such a way that it cannot move from one side of the bathtub to the other. At the same time, it is important that the surface of the bath board is supported securely by the sides of the bath.

Assembly instructions

The legs are mounted with screws

The screw is mounted from the top of the board and tightened from underneath

The handle is tightened with 2 screws from underneath up through the board into the handle.

Accessories

No accessories are available for the product.

Transport recommendations

There are no special considerations to take when transporting the bath mat other than general caution.

4. Technical Data:

| Bath board | 302034 |
|-------------------|------------------------|
| Outer dimensions | 35 x 75,7 x 18,3 cm |
| For bathtubs with | Innenbreite 38 - 64 cm |
| Weight | 2,35 kg |
| Max. load | 150 kg |

Product label

The product label is located on the backside of the board

Image of the product label

| | | | |
|--|-------------------------------|--|--------------------|
| | QR-code link to homepage | | Manufacturing date |
| | Manufacturer | | Loading force |
| | Observe the user manual | | Max width |
| | To be used indoor and outdoor | | serial no |
| | CE marked product | | Item Number |
| | User weight | | Medical Device |
| | Only to be used indoor | | |

Overview icon meaning

5. Care & Maintenance

Normal dirt on metal and plastic parts can be removed with standard cleaning agents. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (no solvents or abrasives). Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional.

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary

Recommended service life: Normally 10 years depending on use.

Storage

We recommend storage in a dry place.

Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

6. Most common problems and solutions

| Symptoms | possible cause | what to do? |
|--|--|--|
| The bath board sits loosely on the bathtub | The legs of the bath are not tightened enough and are not close enough to the edge of the bath | Adjust the bene and retighten the legs |

7. Disposal of the product

The device cannot be deposited with normal garbage. It must be delivered to the public station for re-use.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly.

The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

PRODUCT LIST

Shower board 302034

ACCESSORIES LIST

| Item no. | Accessories item no. |
|----------|----------------------|
| | |

Harmonized norms used during conformity estimation:

PN-EN ISO 14971:2012, PN-EN 12182:2012, EN 581-2:2009, EN 1022:2005, PN-EN 1041:2009

In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted

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