SELF-STANDING SUPPORT RAIL

Art. 302090

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

The Self-standing support rail is a medical device intended only to be used by persons who need support when sitting down or when standing up while using the toilet. The device is suitable for persons who as a result of various diseases need a stable support to get up and/or sit down easily. The device should be used inside bathrooms only.

Indications

The device is indicated for people with functional limitations, people in wheelchairs and other people with limited mobility of the lumbar spine, knees or hips, and for all those who have difficulties performing everyday activities on the toilet. Any use other than stated above is prohibited.

Contraindications

The device is not indicated for those patients who can't maintain proper body balance in a sitting and standing position. The device must not be used if the user has impaired perception. Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
---	--

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- The device must only be used on stable surface.
- Use caution when assembling or adjusting any parts of the device.
- Make sure that all the legs are set to the same height.
- Use device only on flat smooth surface with all 4 legs stably placed on the surface.
- Always check stability of the device before use.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 2 Side frames;
- ✓ 4 Legs;
- ✓ 2 Adjustable front bars;
- ✓ 8 Locking Clips;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

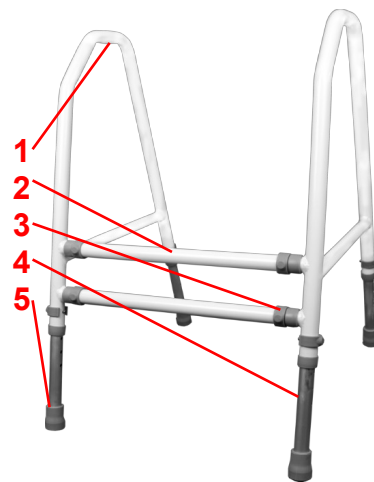
3. Description

Product description

The Self-standing support rail is designed for individuals who require support during standing up or sitting down. The frame is made of aluminium tubing that makes it corrosion resistant including the height-adjustable legs. No tools are required for assembly.

The device consists of the following elements:

1. Side handle
2. Adjustable front bar
3. Locking clip
4. Height-adjustable leg
5. Non-slip rubber shoe

Assembly description

Insert 4 legs to the side frames' bottom tubes and lock them at the desired height with the clips provided. Make sure that all legs are set to the same height and that they are securely locked. The two front bars can now be attached by sliding them onto the short, horizontal tubes located on the side frames, and locking them with the remaining clips. Check that all the legs are firm and stable by placing the rail on a flat surface. It is possible to adjust the width of the self-standing support rail with the locking clips.

Function description

Adjusting the height and width of the self standing support rail

To adjust the height of the rail, all legs must be adjusted to the desired length. To do so take the locking clip out of the leg, slide the leg to the desired setting and put the locking clip back in the opening. Repeat for the other legs and make sure all four legs are adjusted to the same setting.

To adjust the width of the rail, both front bars must be adjusted to the desired setting. To do so, on one side take both locking clips out of the front bars, slide the bars to the desired setting and put locking clips back in the openings. Repeat for the other side and make sure both front bars are adjusted to the same setting on each side.

WARNING! Don't adjust the height and width during use and always make sure all locking clips are mounted correctly.

Accessories

There are no accessories provided for this device.

4. Technical data

Self-standing support rail	Art. 302090
Dimensions (W x D x H)	46-52 x 53-61 x 61-78 cm
Inner width	44 - 49 cm
Side Handles height	61,6 - 78,6 cm
Max. user weight	150 kg
Product weight	1.88 kg

Product label

Example of the product label

	QR-code link to homepage		CE marked product
	Manufacturer		Manufacturing date
	Observe the user manual		Max load
	Serial no.		Medical Device
	Item Number		Max width
	Only to be used indoor		

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. The entire product can be disinfected with a commercially available disinfectant. Always follow the disinfectant manufacturer's instructions for use and safety when disinfecting.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

<i>Problems</i>	<i>Possible causes</i>	<i>Solutions</i>
The Self-standing support rail is not stable	Legs are adjusted to different heights	Adjust all legs to the same height

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

mobilex.dk/medias/2022-01/ce_dokumentation_fritstaaende_toiletstoette__302090__9254.pdf

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