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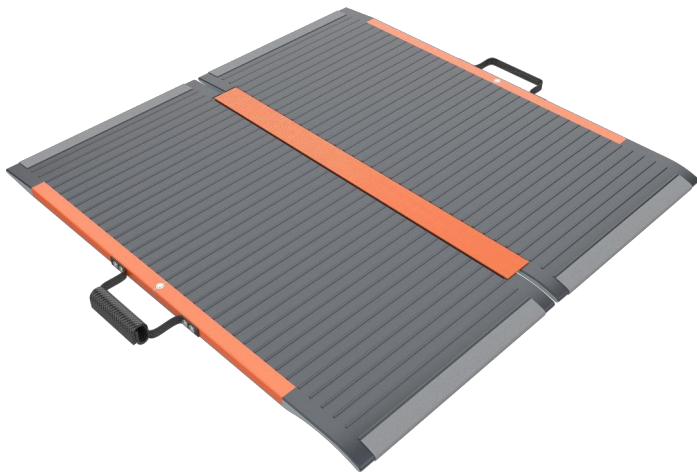
1. Introduction

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

Intended use of the device

The Mobilex CB Ramp is medical devices which is intended to be used only to assist patients in wheelchairs or scooters. They enable the passage of elevated points (e.g. Stairs) with or without the assistance of a helper. Mobilex Ramps thereby enables patients, sitting in a wheelchair because of invalidity, to enter or leave buildings or cars, buses etc. The ramps can be used as well inside as outside. Any other use is prohibited

By questions please take contact to your dealer or directly to Mobilex A/S.

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 E-Mail: info@mobilex.dk
--	---

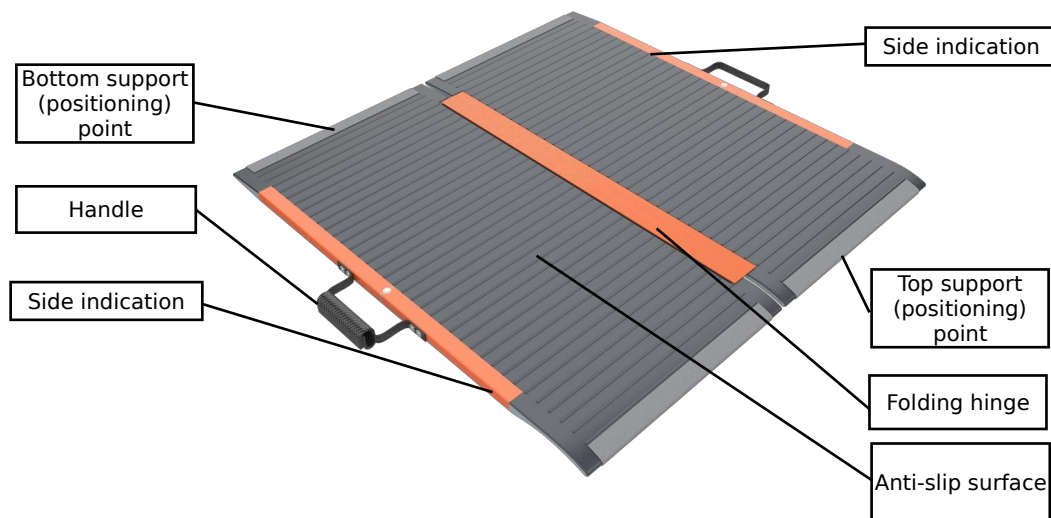
2. Safety requirements

- Make sure that this user manual is read by all persons using the device. Manufacturer doesn't take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- During different adjustments which are possible on the device the user must take care not to squeeze the fingers or other body parts between two moving elements of the device
- The device must only be used on a stable surface.
- Ramps must not be loaded with more than the "max load" described in Technical Data and on the label.
- Be careful during folding or unfolding of the ramp. There is a risk of finger stuck or cut between parts of the ramp.

3. Description

Product description

Mobilex CB-Ramp is foldable platforms made of carbon fiber. The surface is covered with special material for anti-slipping purposes. THE CB-Ramp is supplied in one carton.

The content is:

- ✓ 1 Carbon ramp
- ✓ 1 user manual

Transport recommendations

The CB ramp can be folded for transport and locked. Always assure that ramp during transportation is safely secured against unintended movements.

Accessories

There are no accessories provided for CB ramp.

4. Technical data

Mobilex CB Ramps	CB-082
Length	82 cm
Width	78 cm
Weight	3 kg
Max. load	350 kg
Folded size	82 x 39 cm

Product label

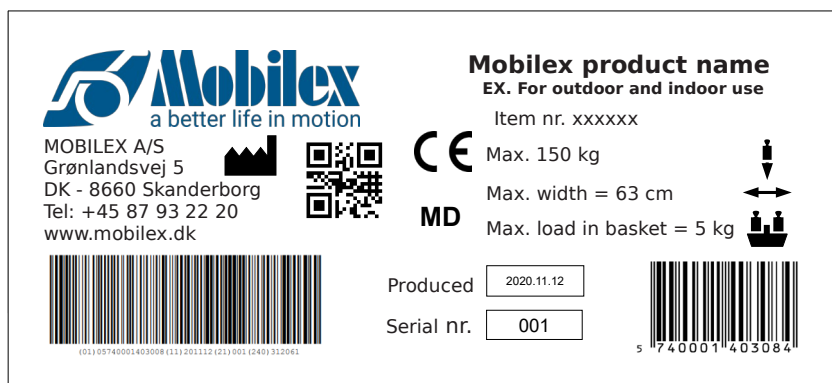
Image of the product label (not original)

5. Maintenance and care

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary

Recommended service life: Normally 10 years depending on use.

Warning

 Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

6. Reuse

The product is suitable for recycling. Before recycling, where a new user is to take over the product, the product must be checked according to point 5.

7. Most common problems and solutions

<i>Symptoms</i>	<i>possible cause</i>	<i>what to do?</i>
Ramp is not stable	The surface under the ramp is not flat	Check the position of the ramp. Put it on flat, smooth surface
Ramp is not stable	Ramp is putted on the surface upside down	Check the position of the ramp.

8. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

9. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,