## FOREARM CRUTCH

Art. 315021

**Gebrauchsanleitung**  
**Brugervejledning**  
**User Manual**  
**Manuale**  
**Bruksanvisning**  
**Instrukcja Obsługi**  
**Manuel de l'utilisateur**

Soft grip

Stable forearm support

Durable design

**CE**

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## 1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

### Intended use of the device

Forearm crutches are designed for use by people with limited mobility. They are used as an orthopaedic aid to support balance, walking, sitting and standing. The product is designed for indoor and outdoor use.

### Indications

The product is intended for the elderly, people with limited mobility of the lumbar region, knees or hips and for all those who have difficulties with standing and walking without an additional support point. The user is required to be able to balance independently while sitting and standing, as well as have adequate ability to move independently.

### Contraindications

Do not use the product if you have problems with balance or perception or if you have insufficient muscle strength to maintain a standing position. The choice of an orthopedic aid adapted to the patient's condition and needs must always be made by a medical specialist or physiotherapist.

### Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

### Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: Telefax: Email:	+45 87 93 22 20 +45 87 93 17 77 info@mobilex.dk
---	------------------------------	---

## 2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- The product should be completely assembled, adjusted in height and secured before use.
- Use caution when assembling or adjusting any parts of the device.
- Always check that the product is rigid and well tightened before each use.
- Use extreme caution with the forearm crutch on stairs and slippery surfaces.
- Clean the product regularly.
- Always check the stability of the forearm crutch before use.
- The rubber feet must not be excessively worn.

### Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Forearm crutch;
- ✓ 1 User manual;

### Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

## 3. Description

### Product description

The forearm crutch gives you a sense of security and fits very well in your hand thanks to the handle being made of soft-touch TPE. Since the forearm crutch is adjustable in length, it can be adjusted to the user's height.

### Assembly description

This device doesn't require an assembly instruction.

### Function description

#### Length adjustment

To adjust the length of the forearm crutch, first loosen the retaining ring by turning it counterclockwise. Once the ring is loose, fully press the locking button and slide the lower tube so that the button pops into a hole corresponding to the desired length. Lastly re-tighten the retaining ring.

**WARNING!** After adjustment and before use, make sure the retaining ring is tightened so that the unit is stable and there is no unintended movement. Failure to comply with this warning may result in personal injury.

## Accessories

There are no accessories provided for this device.

## 4. Technical data

Forearm crutch	Art. 315021
Length	93-122 cm
Weight	0,52 kg
Max. user weight	150 kg

## Product label

Example of the product label

## 5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. For disinfection, use only disinfectants certified for use with medical devices available locally.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

**Recommended service life:** Normally 10 years, depending on use.

**Storage:** We recommend storage in a dry place in temperatures above 0 °C.

## Transportation

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

## Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

## 6. Most common problems and solutions

Problems	Possible causes	Solutions
When used the forearm crutch squeaks	The threaded bush is not tightened enough	Tighten the threaded bush

## 7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

## 8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

## 9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

[mobilex.dk/medias/2021-11/ce\\_albuestok\\_med\\_bloedt\\_haandstoette\\_315021\\_8917.pdf](http://mobilex.dk/medias/2021-11/ce_albuestok_med_bloedt_haandstoette_315021_8917.pdf)

or via the QR code on the right.

	QR-code link to homepage		Manufacturer
	CE marked product		Item Number
	Observe the user manual		Max load
	Medical Device		Manufacturing date
	To be used indoor and outdoor		Serial no

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