SEAL WHEELCHAIR

Art. 273040, 273045, 273050, 274040, 274045, 274050, 275040, 275045,
273140, 273145, 273150

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

The intended use of the wheelchair is to allow the user to move around in a sitting position, independently or with the help of an attendant. Using the wheelchair for purposes other than those indicated above, is prohibited.

Indications

Manual wheelchairs are indicated for people with mobility limitations who have problems with walking, standing or sitting without stable support.

Contraindications

The wheelchair should not be used by people who cannot maintain a stable sitting position. Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: Telefax: Email:	+45 87 93 22 20 +45 87 93 17 77 info@mobilex.dk
---	------------------------------	---

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- The device must only be used on stable surface.
- When adjusting the device the user must take care not to squeeze the fingers or other body parts between components.
- Ensure that all 4 wheels are always in contact with the surface during use. This will ensure proper wheelchair balance and will help avoid accidents.
- Using the wheelchair after exposing it to direct and prolonged contact with the sun may lead to skin burns.
- It is not recommended to use the wheelchair on the beach, in mud or in extreme weather conditions.
- The attendant of the wheelchair user should be physically capable and completely healthy.
- Avoid leaving the wheelchair near sources of heat and open flames (campfires, stoves). Smoking is also not recommended during use as the wheelchair is not fire resistant.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Wheelchair;
- ✓ 2 Footrests;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

“Seal” is a simple yet solid wheelchair. The device has just enough functions to keep the usage comfortable and reliable without raising its complexity.

The device consists of:

1. Upper backrest
2. Armrest
3. Lower backrest
4. Seat
5. Footrest
6. Leg support belt
7. Footrest detaching lever
8. Footplate height adjustment bolt
9. Footplate
10. Push handle
11. Lever for folding the seat for transport
12. 24" rear pneumatic wheel
13. Push rim
14. Parking brake handle
15. Front turning wheel
16. Quick release axle
17. Tipping tube

The 12" version differs in following components:

18. Parking brake (for 12" wheel version)
19. 12" rear pneumatic wheels

The remaining components remain the same.


24" wheel version and quick-release axles

12" wheel version

Assembly description

The only elements that require assembly are the footrests. Insert them according to the **Attaching/detaching footrests** section in **Function description** below.


Function description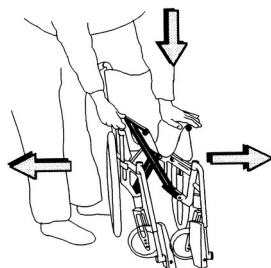

Folding/unfolding the frame

To unfold the wheelchair, hold the seat tubes and push them apart outward, then push down on both of them (left image) until the seat upholstery is flat.

To fold the wheelchair, pull the seat upholstery up and guide one seat tube inwards (right image) until the wheelchair is folded.

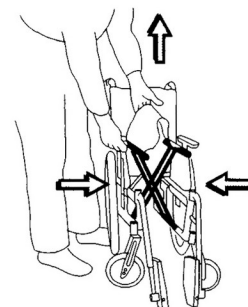Lifting/lowering armrests

To flip an armrest back, press the locking button on the front of the armrest and rotate it towards the back of the wheelchair. To attach the armrest back in its place, rotate it towards the front of the wheelchair and fit the front of the armrest into its attachment point. Lightly push the armrest down so it locks.

Usage of parking brakes

To engage the parking brakes, push the parking brake levers forwards (24" wheel version), or pull the brake levers up (12" wheel version), on both sides of the wheelchair. To release the brakes, pull the levers backwards (24" wheel version) or push the brake levers downwards (12" wheel version), on both sides.

Usage of attendant brakes (optional)

To slow down the wheelchair, pull both brake handles up evenly to control the braking force. For full braking, pull the handles all the way up. To disengage the brakes, simply release the brake handles. You can use the brakes to maneuver the wheelchair by activating the brake on the side you want to turn to.

The attendant brake handles also have a parking brake function. To engage the parking brake function, pull the brake handles all the way up and lock the small black lever under the handle by pushing it down. To release the parking brake pull the small black lever back up.

WARNING! The parking brakes are not designed as driving brakes and only serve to lock the wheelchair in place (parking). They may not be used for braking while driving!

Usage of quick release axles (option for 24" wheel ver.)

To detach a wheel using a quick release axle, press and hold the button in the middle of the wheel, and pull it out of the frame. To attach it back to the frame, press and hold the button. Fit the axle into its slot and fully push the wheel into the frame. Release the button.

Attaching/detaching footrests

To attach a footrest, fit the top and bottom attachment points of the footrest to the ones on the frame from the side of it, and turn the footrest inwards until it locks in place. To detach a footrest, pull the footrest detaching lever towards the wheelchair and turn it to either the outside or inside of the wheelchair and pull it up.

The footrests can be folded inwards for transport.

Adjusting footplates

To adjust the height of the footplate, loosen the height adjustment nut and move the footplate to the desired height. Tighten the adjustment nut and repeat the adjustment for the other footplate.

Tip-pedal

The tip-pedal on wheelchairs makes it easier for the attendant to avoid obstacles by tilting the wheelchair backwards (left image). The tip-pedal is built into the frame (right image).

NOTE: DO NOT tilt the wheelchair with the handles, but use the tip-pedal!

Accessories

The device can be equipped with the following accessories (not a part of the standard equipment):


- ✓ Anti-tip

4. Technical data

Seal wheelchair	Art. 273040	Art. 273045	Art. 273050	Art. 273140	Art. 273145	Art. 273150
Seat width	40 cm	45 cm	50 cm	40 cm	45 cm	50 cm
Quick release axle	Yes					
Attendant brakes	No			Yes		
Total width	58 cm	63 cm	68 cm	58 cm	63 cm	68 cm
Weight NW/GW kg	18/20.5	18.5/21	20/22.5	18/20.5	18.5/21	20/22.5
Tires front/back	8"x1" PU/24"PU					
Max user weight	120 kg	120 kg	135 kg	120 kg	120 kg	135 kg

Seal wheelchair	Art. 274040	Art. 274045	Art. 274050	Art. 275040	Art. 275045
Seat width	40 cm	45 cm	50 cm	40 cm	45 cm
Quick release axle	No				
Attendant brakes	No				
Total width	58 cm	63 cm	68 cm	55 cm	60 cm
Weight NW/GW kg	18/20.5	18.5/21	20/22.5	15/17.5	16/18
Tires front/back	8"x1" PU/24"PU			8"x1" PU/12"PU	
Max user weight	120 kg	120 kg	135 kg	100 kg	110 kg

Product label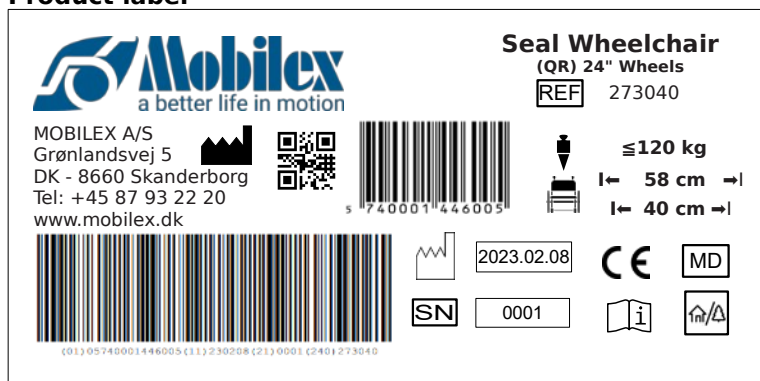

Example of the product label

	QR-code link to homepage		Manufacturer
	CE marked product		Item Number
	Observe the user manual		Max load
	Medical Device		Manufacturing date
	To be used indoor and outdoor		Serial no.
	Max width		

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. For disinfection, use only disinfectants certified for use with medical devices available locally.

Wheels can be cleaned with hot water with a mild cleaning agent. After cleaning the wheel bearings should be lubricated with oil periodically or when the wheels start squeaking.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:


- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

It is recommended to transport the device folded, in a car trunk. If that method of transportation is unavailable, the folded wheelchair can be transported behind the front seats of the car (right image).

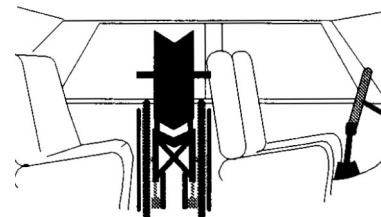Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

Problems	Solutions
The wheelchair seems to tilt to one side	<ul style="list-style-type: none"> • inflate tires • check fastening of the front turning wheels • one of the rear tires might be inflated more than the other one
The wheelchair is hard to push	<ul style="list-style-type: none"> • inflate tires • check fastening of the front turning wheels • remove dirt or entangled hair from the front turning wheels • too large load applied on the front turning wheels: move the center of gravity
The wheelchair is hard to turn	<ul style="list-style-type: none"> • inflate tires • ensure, that bolts fastening the front turning wheels are not tightened too strong • remove dirt or entangled hair from the front turning wheels
Brakes are not working correctly	<ul style="list-style-type: none"> • inflate tires • adjust the distance between the brake and tires
It is hard to fold and unfold the wheelchair	<ul style="list-style-type: none"> • the upholstery fitted too tight • clean up or apply grease to the cross under the seat
The wheelchair is not stable	<ul style="list-style-type: none"> • inflate tires • ensure, that all screws/bolts and nuts are tightened • ensure, the wheelchair is properly adapted to the passenger

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at https://mobilex.dk/medias/2023-03/ce_seal_11708.pdf or via the QR code on the right.

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