## WALL MOUNTED SHOWER CHAIR

Art. 302040

**Gebrauchsanleitung**  
**Brugervejledning**  
**User Manual**  
**Manuale**  
**Bruksanvisning**  
**Instrukcja Obsługi**  
**Manuel de l'utilisateur**

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## 1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

### Intended use of the device

Wall mounted shower chair is medical device intended only to be used by patients that need support during washing under the shower. The wall mounted shower chair is suitable for persons who as a result of various diseases need a stable support to get easy access to all human body parts during washing. The device should be used inside bathrooms only. The chair can be maximally loaded with 100 kg. Any use other than stated above is prohibited.

### Indications

The device is indicated for people with functional limitations, people in wheelchairs and other people with limited mobility of the lumbar spine, knees or hips, and for all those who have difficulties performing everyday personal hygiene in standing position. Using the device for purposes other than those indicated above, is prohibited.

### Contraindications

The device is not indicated for those patients who can't maintain proper body balance in a sitting and standing position. The device must not be used if the user has problems with impaired perception. Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

### Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

### Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
---	--

## 2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- Make sure the device is securely attached to a suitable sturdy wall before use.
- Make sure both feet are set to the same length and that they are placed on a flat surface.
- Use caution when unfolding the device to not squeeze any body parts between components of the device.

### Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Wall mounted shower chair;
- ✓ 1 Base plate;
- ✓ 1 Plastic bag with 6 x plastic caps;
- ✓ 1 User manual;

### Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

## 3. Description

### Product description

The wall-mounted shower chair is made of corrosion-resistant materials and is protected against rust. The seat of the shower chair has height-adjustable legs and feet, and is fixed to the wall plate. This allows the shower chair to be locked in place both in the sitting position and folded up, saving space.

**The device consists of the following elements:**

1. Mounting plate
2. Seat
3. Folding tube
4. Leg length adjustment clip
5. Foot

**Assembly description**

The shower chair should be secured to a wall capable of supporting the vertical load and leverage affected upon it by the user. The distance from the floor to the seat should match that of a regular chair that the user comfortably sits on with the feet on the ground. The legs and feet should be adjusted accordingly (described in **Function description**).

Take 6 bolts, washers and wall plugs (not included). Fit the mounting plate and mark the position of the six mounting holes on the wall, make sure they are leveled with the ground. Drill the holes in the wall and insert the wall plugs. Put the washers between the plate and the bolts, and screw the mounting plate to the wall tightly. The chair can now be fixed to the mounting plate with the supplied nuts and washers attached to the mounting plate. Check that the chair is correctly mounted by folding and unfolding it a few times.

**NOTE:** All operations need to be carried out by a capable person. No mounting tools are provided. The choice of fastening material depends on the condition and types of the wall.

**Function description**

**Adjusting leg length**

To adjust the length of a leg, take the leg length adjustment clip out of the leg, slide the leg to the desired setting (in steps of 2,5 cm) and put the leg length adjusting clip back in the opening. Repeat for the other leg and make sure both legs are adjusted to the same setting.

**Adjusting foot height**

To adjust the height of a foot, loosen the locking nut, rotate the foot so that it goes to the desired setting and tighten the locking nut.

**Folding the chair**

Grab the front edge of the seat and pull it upwards until it is fully folded. To unfold the chair, pull the same edge of the seat away from the wall until the legs touch the floor. The legs deploy automatically.

**WARNING:** Only sit in the center of the seat.

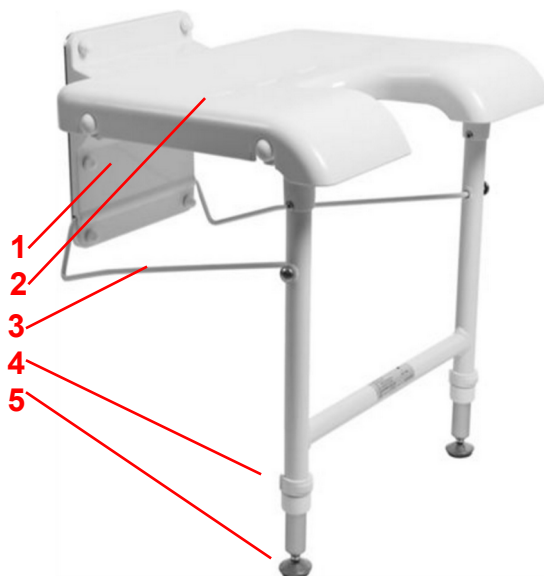
**NOTE:** The chair can be folded up any time it is not in use to save space.

**Accessories**


There are no accessories provided for this device.

**4. Technical data**

Wall mounted shower chair	Art. 302040
Width	45 cm
Length (depth)	45,5 cm
Seat width	44 cm
Seat depth	37 cm
Seat height (adjustable)	43.5 cm - 51 cm, in 2.5 cm increments
Weight	3,5 kg
Max. user weight	100 kg

## Product label

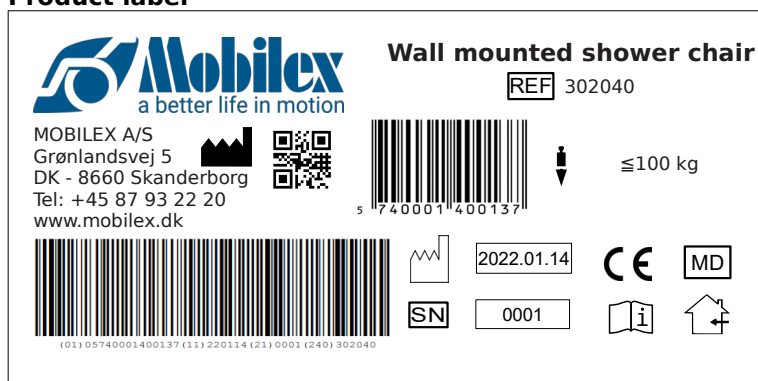Example of the product label

	QR-code link to homepage		Manufacturer
	CE marked product		Item Number
	Observe the user manual		Max load
	Medical Device		Manufacturing date
	To be used indoor		Serial no

Meaning of the icons

## 5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. For disinfection, use only disinfectants certified for use with medical devices available locally.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the product for bends, cracks, damage, wear or corrosion
- ✓ General re-tightening of the components

**Recommended service life:** Normally 5 years, depending on use.

**Storage:** We recommend storage in a dry place in temperatures above 0 °C.

## Transportation

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

## Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

## 6. Most common problems and solutions

Problems	Possible causes	Solutions
The chair isn't stable	Legs adjusted to different heights	Readjust legs to the same height
	The chair is mounted loosely	Check the wall assembly and tie all screws.
The seat isn't in horizontal position	Legs adjusted to wrong length: - too short (seat falls forward) - too long (seat reclined to the back)	Adjust legs to the length, that allows the seat to lie in horizontal position
The seat isn't level with the floor	The chair is not mounted in parallel to the floor	The chair needs to be mounted to the wall correctly
		The feet can be adjusted to account for slight tilting according to the Function description section

## 7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

## 8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

## 9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at

[mobilex.dk/medias/2021-08/ce\\_vaegmonteret\\_badestol\\_302040\\_8313.pdf](http://mobilex.dk/medias/2021-08/ce_vaegmonteret_badestol_302040_8313.pdf)

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