TR & LR RAMPS

"TELESCOPIC"

Art. TR-150, TR-210, TR-300 & LR-150, LR-210

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

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mobilex.dk/medias/2021-11/ce_rampe_tr_150_tr_2_10_tr_300_lr_150_lr_210_8873.pdf

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

To be used to assist people in wheelchairs or mobility scooters by enabling passage over elevated points.

Indications

Ramps are indicated for people with mobility limitations who due to problems with walking, standing or sitting are forced to use mobility devices such as wheelchairs, mobility scooters, rollators etc. Ramps are indicated for elimination of architectural obstacles by smooth connection of the level differences that a wheelchair user has to overcome and thus they allow the wheelchair user to reach places normally inaccessible to him/her. The ramp is intended for use independently by the user or with the help of an attendant. Using the ramp for purposes other than those indicated above, is prohibited.

Contraindications

Contraindications cannot be tied to a specific clinical picture. The choice of a suitable ramp type and length depends on multiple factors and must always be made with the help of a specialist doctor or physiotherapist. Observe the safety information in chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
---	--

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- Use caution when assembling or adjusting any parts of the device.
- The device must only be used on a stable surface.
- Be careful during folding or unfolding of the ramp. There is a risk of fingers getting stuck or cut between parts of the ramp.
- The ramp can not be used at angles larger than indicated in Technical data.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 2 Ramps;
- ✓ 1 Carrying bag (LR Versions only);
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

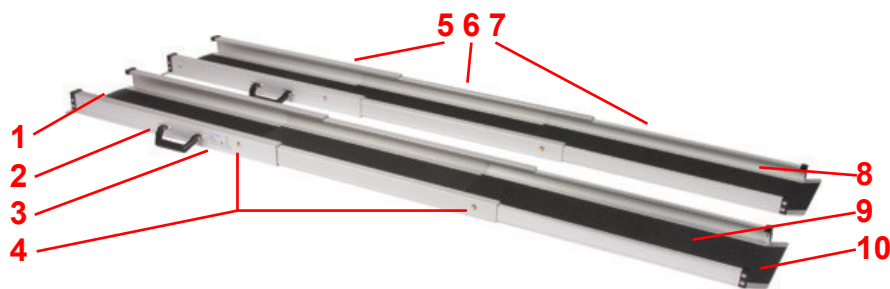
3. Description

Product description

TR/LR-Ramp set consists of 2 separate rails made of aluminium. The surface is covered with special anti slipping material.

The device consists of:

1. Top support
2. Carrying handle
3. Product label
4. Spring lock release buttons
5. Wider segment
6. Medium segment
7. Narrow segment
8. Side protection
9. Anti-slip surface of aluminium silicate
10. Bottom support

Assembly description

This device doesn't require an assembly instruction.

Function description

Extending the ramp

To extend a ramp rail, press the spring lock release button on the wide segment and slide the medium segment out of it until it automatically locks in place. Repeat for the small segment. Repeat the whole extension process with the second ramp rail.

To push a ramp rail together, press the spring lock release buttons and push the small, medium, and large segments together in that order.

WARNING! The segments can completely fall out if you press the button and slide them beyond the lock.

Positioning the ramp

Place the top support on the elevated surface, and the bottom support on the ground level. Position the second ramp rail in parallel to the first one. Make sure the rails are steady and placed on a stable surface before use.

Accessories

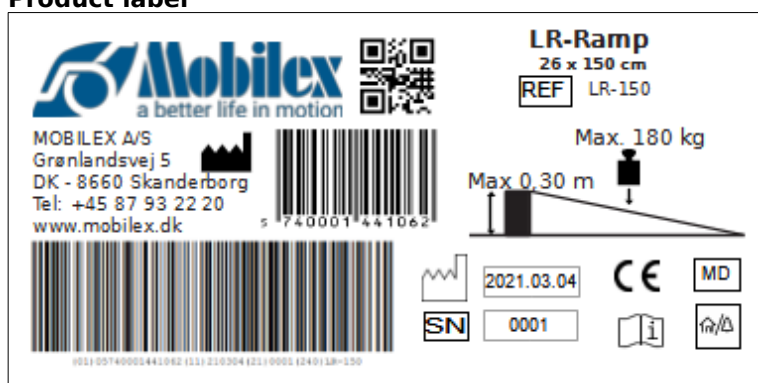
The TR-ramps can be equipped with the following accessories (not a part of the standard equipment):

- ✓ Carrying case

4. Technical data

TR-Ramp / LR-Ramp	TR-150	TR-210	TR-300	LR-150	LR-210
Length (extended)	150 cm	210 cm	300 cm	150 cm	210 cm
Width (per rail)	26 cm	26 cm	26 cm	16 cm	16 cm
Inner track width (per rail)	19 cm	19 cm	19 cm	10 cm	10 cm
Height of side protection	7 / 5 cm	7 / 5 cm	7 / 5 cm	5,5 / 3 cm	5,5 / 3 cm
Max. Elevation / height	12° / 30 cm	12° / 42 cm	12° / 60 cm	12° / 30 cm	12° / 42 cm
Weight	2 x 5,5 kg	2 x 7,5 kg	2 x 9,5 kg	2 x 3 kg	2 x 4 kg
Max. load (per pair)	225 kg	225 kg	225 kg	180 kg	180 kg
Pushed together dimensions (L x W x H)	74 x 26,5 x 7	95 x 26,5 x 7	125 x 26,5 x 7	74 x 16,5 x 5,5	95 x 6,5 x 5,5

Product label

Example of the product label

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Batch no/ serial no
	CE-marked product		Item Number
	To be used indoor and outdoor		Medical Device
	Max Elevation / height		

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. For disinfection, use only disinfectants certified for use with medical devices available locally.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the product for bends, cracks, damage, wear or corrosion
- ✓ Wear parts are checked and replaced if necessary

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

It is recommended to fold the device for transportation. The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

<i>Problems</i>	<i>Possible causes</i>	<i>Solutions</i>
Ramp is not stable	The surface under the ramp is not flat	Check the position of the ramp. Put it on a flat, smooth surface
	Ramp is placed upside down	Check if the ramp is positioned correctly

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at

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