BOA Air Pump

Cordless
Art. 260020

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

Portable and convenient air pump

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1. Introduction

Thank you for buying a high-quality product. These instructions of use contain a description and important information about the use of the product. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

In case of any unwanted events related to the device, please contact:

Authorised Representative and Importer:	Distributor:
Levabo Sverigesvej 20A 8660 Skanderborg Denmark	Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark
Phone: +45 31 70 40 25 Email: info@levabo.dk	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Only use the product for the described purpose.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- This device can be used by children aged from 8 years and above, and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge on the device, given they have been given supervision or instruction concerning safe use of the appliance and understand the hazards involved.
- Never allow children to play with the device or the accessories.
- Cleaning and user maintenance should not be done by children without supervision.
- Never allow children to connect/disconnect the air pump to/from power sources.
- The air pump must not operate for more than 10 minutes at a time, after which it must be switched off and put aside for cooling for at least 10 minutes before restarting. Inflation over a long period will overheat the air pump and may damage it.
- If the air pump emits abnormal sounds or overheats, turn it off immediately and let it cool down for about 30 minutes.
- Check the air pump and battery before use. An expert should repair damaged parts. Incorrect use of the air pump may damage it.
- Never expose the air pump or battery to rain, frost or temperatures above 50° C. Allow batteries to cool down completely before charging. Never charge the battery in humid or wet locations.
- Never use the air pump near flammable fluids or gases. Make sure your working area is well lit.
- Never pump air into the mouth, ears, nose or any other body part of people or animals. Never use the air pump as a toy.
- Regularly check the cord of the charger. Never carry the charger by the cord and never pull the plug out of the outlet by the cord. Make sure that the cord is not exposed to hot objects, oil or sharp edges. Pull the charger plug out of the plug socket when the charger is not in use.
- Never charge the battery outdoors.
- The battery and charger are designed to be used only with each other. Only use the original enclosed charger for charging.
- Protect yourself against electric shock. Avoid physical contact with grounded surfaces such as pipes, radiators, electric stoves, refrigerators, etc.
- If the pump is being disposed of, do not open the battery, only open the pump and remove the battery.
- Keep the air pump clean to achieve maximum output and safety. Examine the plug, charger and cord at regular intervals and have them repaired by an expert if they are damaged.
- Protect the pump and battery from impact. The pump should be stored in a dry, frost-free place.
- Store the air pump in a secure location. If the air pump is not used for long periods of time, it should be stored in a dry, locked space inaccessible to children.
- Protect the pump against heat and fire: danger of explosion!
- Do not place the pump on a heater and do not expose it to solar radiation for a long period.
- Stay alert. Always pay attention when using the air pump.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Air pump;
- ✓ 3 Adapter pcs. inside pump on the bottom cover plate;
- ✓ 1 Adapter hose;
- ✓ 1 Charger;
- ✓ 1 Charge base;
- ✓ 2 Mounting wall plugs with screws;
- ✓ 1 Carrying bag;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

The cordless air pump has been specially designed for simple and efficient inflation. The air pump provides a small quantity of air at a relatively high pressure which means that it is not suitable for inflating large inflatable objects requiring a large airflow at low pressure, such as air-mattresses, inflatable beach toys and inflatable furniture.

The device consists of the following elements:

1. On/off switch
2. Light button
3. Inner valve rubber part
4. Valve connector ring
5. Front light
6. Illuminated pressure gauge 0 - 150 PSI
7. Adapter set - 3 pcs. inside pump on a plastic cover
8. Adapter hose
9. DC Input
10. AC Charger
11. Pump charging base
12. Carrying bag
13. Inner valve adapter

Assembly description

This device doesn't require an assembly instruction.

Function description

General Use

The air pump is equipped with a valve connector ring (4). To inflate an object, loosen the connector ring (counter-clockwise) and push it down over the valve on the object to be inflated - if this cannot be done, fit a suitable adapter. Tighten the connector ring again (clockwise) and turn the pump on. Loosen the connector ring again before removing the air pump from the object or taking out the adapter.

NOTE: Do not run the air pump for more than 10 minutes in continuous operation mode - and after 10 minutes allow it to cool for a minimum of 10 minutes.

Inflating balls

Loosen the valve connector ring (4) and press the threaded end of the ball adapter (5) into the connector ring and tighten the ring again so that the ball adapter is secured. Carefully push the pointed end of the ball-adapter into the ball's valve. Then press the on/off switch (1) and inflate the ball. Once done, turn the pump off by pressing the switch and carefully remove the ball adapter from the ball. Loosen the connector ring again to remove the ball adapter from the connector ring.

Inflating tires

Loosen the connector ring (4) and fit the pump onto the valve. If the pump do not fit the valve, attach a suitable adapter. For bicycle valves with a French valve (typically racing bike), loosen the valve screw only halfway before fitting the bicycle adapter onto the valve. Tighten the connector ring (4) again. Then press the on/off switch (1) and inflate the tire. Loosen the connector ring (4) again before you remove the air pump from the tire valve.

Charging the battery

For optimum use of the cordless air pump, you should ensure that the battery is always charged. Connect the charger to the pump or charge stand, and connect the charger to a 110V / 230V power source. The indicator glows red when charging, green when fully charged. The supplied battery can be recharged at least 500 times.

- Charging time for the first charge using the accompanying AC charger: 5 hours.
- Subsequent charges: 1 - 2 hours — depending on discharge.
- In order to preserve the battery's capacity, it should be occasionally discharged almost fully.
- With correct use, the battery can be recharged at least 500 times

Accessories

There are no accessories provided for this device.

4. Technical data

Boa Air Pump	Maximum pressure	Air flow	Max current	Rated voltage	Li-ion battery capacity	AC charger Input	AC charger output	Pump Size	Bike tire 0 - 75 PSI	Bike tire 0 - 150 PSI
Art. 260020	145 PSI / 10 bar	9L / min	11A	DC 7,2V	2000 mAh	110/230 V	9V - 1.000 mA	5 x 240 mm	30 Sec	90 Sec

5. Care and maintenance

This air pump is maintenance free, having self-lubricating piston. Do not wash pump inside the housing. Cleaning the pump is done with a damp cloth on the outside housing.

For your own safety you should only use accessories and other parts that are specified in the instructions or recommended for use by the manufacturer. Use of accessories and other parts that are not mentioned in the instructions may entail an increased risk for the user of the air pump.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

6. Most common problems and solutions

<i>Problems</i>	<i>Possible causes</i>	<i>Solutions</i>
The battery takes a long time to charge	The charger is plugged in incorrectly	Plug the charger in correctly

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center. The battery must be removed from the appliance before it is scrapped. To take out the battery:

1. The appliance must be disconnected from the supply mains when removing the battery.
2. Open the pump and disconnect the wires at the battery – avoid any connection between wires !!
3. The battery is to be disposed of safely

BOA Products International and your local retailer accept returned used air pumps. Dispose of a defective pump, must be done in an environmentally friendly manner - do not leave it in nature, do not burn it and never open the pump/battery

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. EU / CE compliance declaration

BOA Products International and here for Mobilex A/S declares under general liability that this product complies with the following standards or normative documents: EN 50 144, EN 55 014, EN 55 104 (battery-powered tool) or EN 60 335, EN 60 555, EN 55 022, EN 50 0811, EN 61 000-3-2, EN 61 000-3-3 (battery-powered charger unit) according to the provisions in directives 89/392/EEC, 89/336 EEC from (1/96), 91/368 EEC,

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