ROUND SHOWER STOOL

Art. 302063, 302064

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

Round shower stool

Round compact seat

Easy
height
adjustment

Sturdy
construction

CE

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

The round shower stool is a medical device intended only to be used by patients that need support during washing under the shower. The device is suitable for persons who as a result of various diseases need a stable support to get easy access to all human body parts during washing. The device should be used inside bathrooms only.

Indications

The device is indicated for people with functional limitations, people in wheelchairs and other people with limited mobility of the lumbar spine, knees or hips, and for all those who have difficulties performing everyday personal hygiene in standing position. Any use other than stated above is prohibited.

Contraindications

The device is not indicated for those patients who can't maintain proper body balance in a sitting and standing position. The device must not be used if the user has impaired perception. Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
---	--

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- The device must only be used on stable surface.
- Use caution when assembling or adjusting any parts of the device.
- Make sure that all the legs are set to the same height.
- Use device only on flat smooth surface with all 4 legs stably placed on the surface.
- Always check stability of device before use.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 2 Frame tubes;
- ✓ 4 Legs;
- ✓ 1 White round seat;
- ✓ 5 Screws, 5 metal washers;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

The round shower stool is designed for individuals who require support during showering or while bathing. The shower stool has an aluminium frame with height-adjustable legs and a seat made of white plastic.

The device consists of the following elements:

1. Seat
2. Height-adjustable legs
3. Non-slip rubber shoe

Assembly description

Place the white seat upside down. Fit the frame tubes to the seat and be sure to place one tube crosswise under the other. Insert a screw with the metal washer in the middle hole, then the other 4 screws in the same way in the 4 holes on the underside of the seat and tighten the screws. Slide the legs onto the frame tubes so that the buttons pop out. Check that all the legs are firm and stable by placing the stool on an even surface. Check and adjust the legs to the desired height, making sure that the legs are set to the same height and all are securely locked.

Function description

Adjusting the height of the round shower stool

To adjust the height of the stool you need to adjust length of all legs to the desired setting. To adjust the length of a single leg, press the button on the inner side of the stool and slightly turn the leg. Slide the leg tube to the desired setting and turn it back so that the button will pop out. All legs should be adjusted to the same height.

WARNING! Don't adjust the height of the shower stool when someone is sitting on it.

NOTE: The seat height is correctly adjusted when the user is sitting on the shower stool and both feet fully touch the ground.


Accessories

There are no accessories provided for this device.

4. Technical data

Round shower stool	Art. 302063, 302064
Dimensions of the seat	Ø 33 cm
Seat height adjustment range	35.5 - 53.5 cm
Max. user weight	130 kg
Product weight	1.66 kg

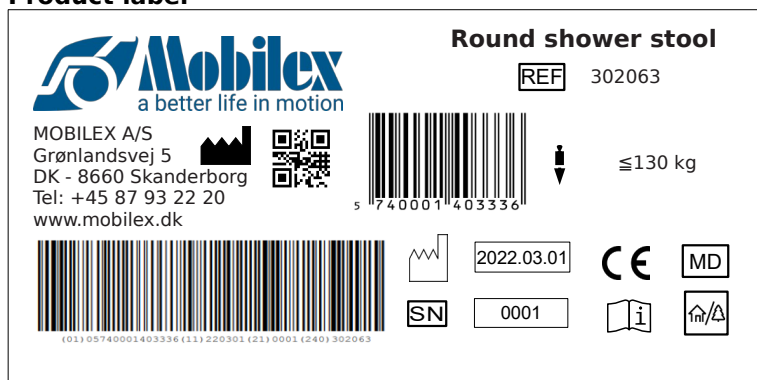Product label

Example of the product label

	QR-code link to homepage		CE marked product
	Manufacturer		Manufacturing date
	Observe the user manual		Max load
	Serial no.		Medical Device
	Item Number		To be used indoor and outdoor

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. The entire product can be disinfected with a commercially available disinfectant. Always follow the disinfectant manufacturer's instructions for use and safety when disinfecting.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

<i>Problems</i>	<i>Possible causes</i>	<i>Solutions</i>
Shower stool is not stable	Legs are adjusted to different heights	Adjust all legs to the same height

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

[mobilex.dk/medias/2021-08/CE%20Badetaburet,%20gr%C3%A5%20med%20hvidt%20%C3%9833%20cm%20s%C3%A6de%20\(302063-6464\)_8343_0x0px.pdf](http://mobilex.dk/medias/2021-08/CE%20Badetaburet,%20gr%C3%A5%20med%20hvidt%20%C3%9833%20cm%20s%C3%A6de%20(302063-6464)_8343_0x0px.pdf)

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