Mobilex "Telescopic" TM ramps

Art. TM-050, TM-150, TM-210 & TM-300

Montage- und Gebrauchsanleitung
Montage- og brugervejledning
User Manual
Manuale d'uso e manutenzione

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1. Introduction

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

By questions please take contact to your dealer or directly to Mobilex A/S.

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 E-Mail: info@mobilex.dk
--	---

2. Safety requirements

- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Be aware of the information on the product label.
- Make sure that this user manual is read by all persons using the ramp. Damages, caused by the fact that the user manual has not been followed, will not be accepted.
- Ramps must not be loaded with more than the max load described in Technical Data and on the label.
- The ramp can be used at maximum inclination of:
TM-055: 11 cm, TM-150: 30 cm, TM-210: 40 cm, TM-300: 60 cm
- Only use the ramp for the purpose it is intended for.
- Avoid making constructive changes on the ramp, unless you have the producers written accept.
- When unfolding the ramp the user must take care not to squeeze the fingers or other body parts.
- The ramp must only be used on a stable surface.

3. Description

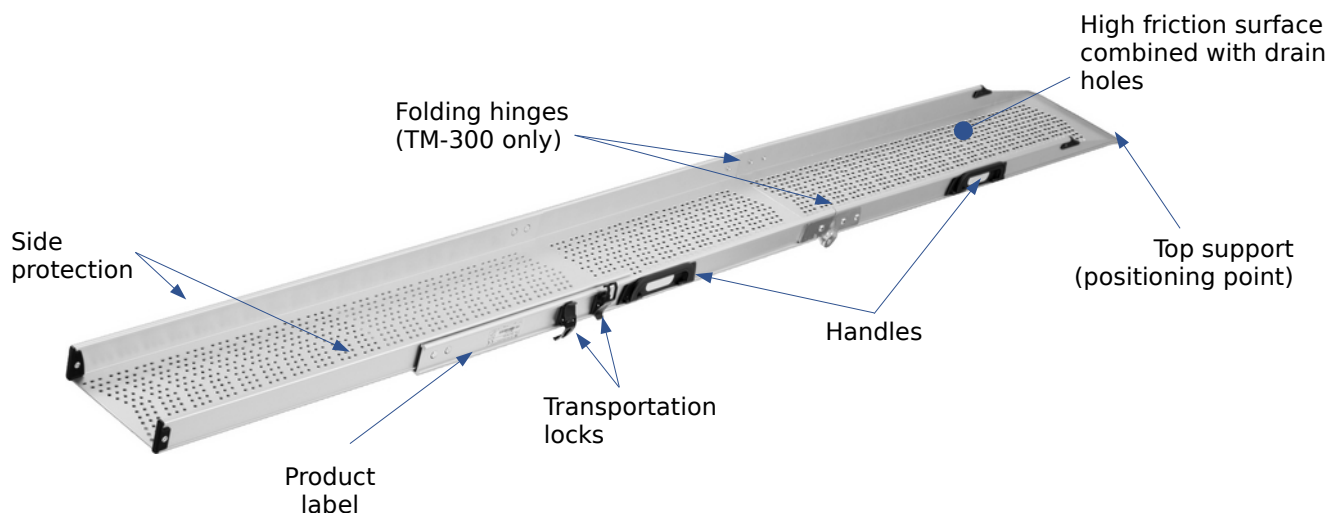
Product description

The TM-Ramp set consists of 2 separate rails made of aluminium with perforated lanes.

Delivery contents

Ramps are supplied in cartons. The contents are:

- ✓ 1 set TM ramps, consisting of 2 of two separate drive-on rails
- ✓ This user manual

Set Up & Function description

- The ramps consist of two separate drive-on rails. The model TM-055 only consists of 2 separate rails, the model TM-150 and TM-210 consists of 2 telescopic segments, model TM-300 has also a further element which is foldable.
- The ramps must be placed on a stable surface.
- Ramp halves must be always placed in parallel position.
- Check that the ramps are stable before driving on them.

Transport recommendations

The model TM-055 only consists of 2 separate rails, the TM-150 and TM-210 ramps have two telescopic elements that can be pushed together for transport and fixed (telescopic ramps). The model TM-300 has an additional element that can be folded. For transport, the transportation locks have to be closed (see picture above).

Accessories

There are no accessories provided for the TM Ramps.

4. Technical data

Mobilex Rampe TR/LR	TR-150	TR-210	TR-300	LR-150	LR-210
Length	150 cm	210 cm	300 cm	150 cm	210 cm
Width (per rail)	26 cm	26 cm	26 cm	16 cm	16 cm
Surface width	19 cm	19 cm	19 cm	11 cm	11 cm
Height of side protect.	7/5 cm	7/5 cm	7/5 cm	5,5/3 cm	5,5/3 cm
Max. Elev. / height	12° / 30 cm	12° / 40 cm	12° / 60 cm	12° / 30 cm	12° / 40 cm
Weight	2 x 5,5 kg	2 x 7,5 kg	2 x 9,5 kg	2 x 3 kg	2 x 4 kg
Max. load per pair	225 kg	225 kg	225 kg	180 kg	180 kg
Folded size cm (LxWxH)	74 x 26,5 x 7	95x26,5x7	125x26,5x7	74x16,5x5,5	95x6,5x5,5

Product label

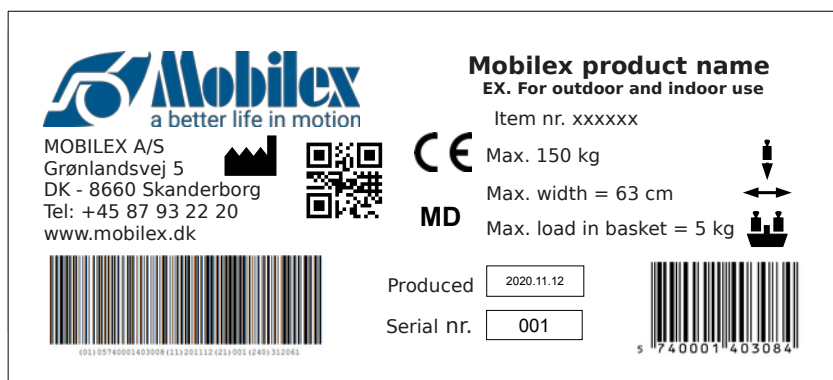
Image of the product label (not original)

5. Maintenance and cleaning

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional

- Lighter dirt and dust can be removed with a wet cloth.
- Heavier types of dirt can be removed with a non-corroding, mild washing agent.
- You can also use garden hose to clean the ramp.

Recommended service life: Normally 10 years depending on use.

Warning

 Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

6. Reuse

The product is suitable for recycling. Before recycling, where a new user is to take over the product, the product must be checked according to point 5.

7. Most common problems and solutions

<i>Symptoms</i>	<i>possible cause</i>	<i>what to do?</i>
Ramp is not stable	The surface under the ramp is not flat	Check the position of the ramp. Put it on flat, smooth surface
Ramp is not stable	Ramp is putted on the surface upside down	Check if the A label is in the upper end and B label in the lower end of the ramp. Correct position if needed.
Ramp doesn't open	The lock strap is not loosen	Loosen the lock strap

8. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

9. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,